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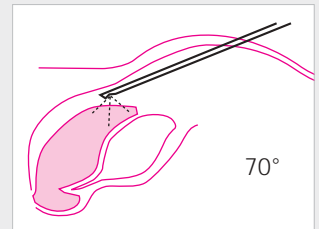
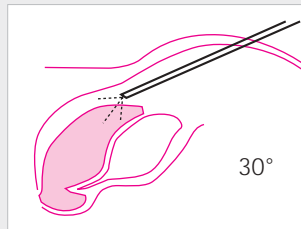
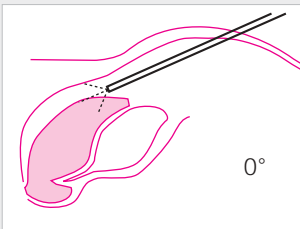
Various directions of view of laparoscopes.




It is very important to have a suitable direction of view for the used telescopes during upper abdominal laparoscopy. The reason is that the working space between the rib cage and anterior surface of the liver is smaller and anatomically more difficult than the pelvic area..

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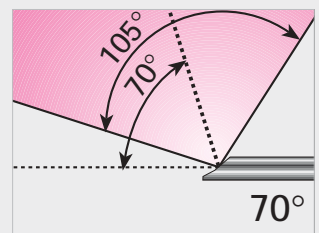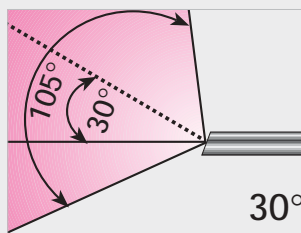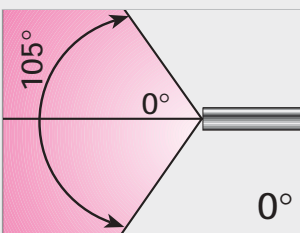As you can see on the documentation it is very important to have a suitable direction of view for the laparoscope. The 0° laparoscope provides orientation and gives an idea about the area located. It allows the more natural approach that permits the surgeon to use the normal perspective. Through rotating the 30° laparoscope a larger area can be observed than with a 0° laparoscope. With the 70° laparoscope the underneath area is well seen, but through the difficult area it can cause problems with the orientation.

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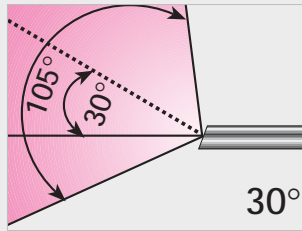
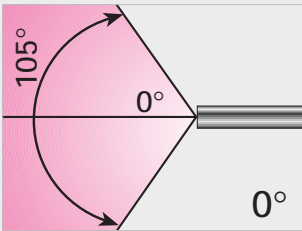
for trocar sizes 10.0 and 11.0 mm,  
for multiple puncture

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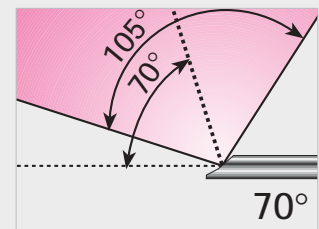
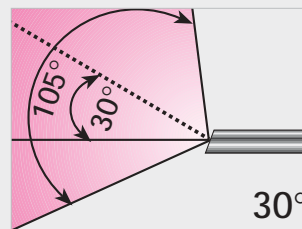
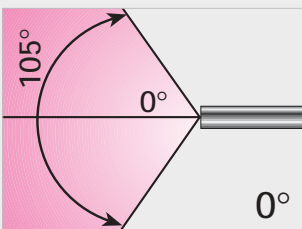
for trocar size 7.0 mm,  
for multiple punctures

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for trocar size 5.0 and 5.5 mm,  
for multiple punctures

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Operating Laparoscope with integrated instrument channel. For trocar size 11.0 mm, for single puncture.

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Outer diameter: 10.0 mm  
total length: 273 mm, working length: 266.5 mm  
Instrument channel: 5.0 mm  
Parallel view, 0°

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Outer diameter: 10.0 mm  
total length: 273 mm, working length: 266.5 mm  
Instrument channel: 6.0 mm  
Parallel view, 0°

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Outer diameter: 10.0 mm  
total length: 273 mm, working length: 266.5 mm  
Instrument channel: 5.0 mm  
Lateral view, 0°

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Outer diameter: 10.0 mm  
total length: 273 mm, working length: 266.5 mm  
Instrument channel: 6.0 mm  
Lateral view, 0°

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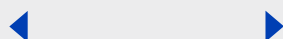
Modular trocar sleeve with multifunctional trap door valve and exchangeable trocar sleeve. With lateral stop cock for gas insufflation. For standard and safety trocars.

5.0 mm	300-020-005	11.0 mm	300-020-011
5.5 mm	300-020-055	12.5 mm	300-020-125
10.0 mm	300-020-010	15.0 mm	300-020-015

Trokarhülsen Trocar sleeves	1 komplettes Set an Dichtungen complete set of gaskets	2 Dichtungskappe sealing cap	Farbe colour
ø	in Packungen zu 10 St., in packages of 10 pcs.	in Packungen zu 10 St., in packages of 10 pcs.	
5.0 mm	300-024-905	300-025-905	weiss/white
5.5 mm	300-024-955	300-025-955	grün/green
10.0 mm	300-024-910	300-025-910	violett/violet
11.0 mm	300-024-911	300-025-911	blau/blue
12.5 mm	300-024-912	300-025-912	rot/red
15.0 mm	300-024-915	300-025-915	grau/grey

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
Modular trocar sleeve with trap door and exchangeable trocar sleeve. For standard and safety trocars.

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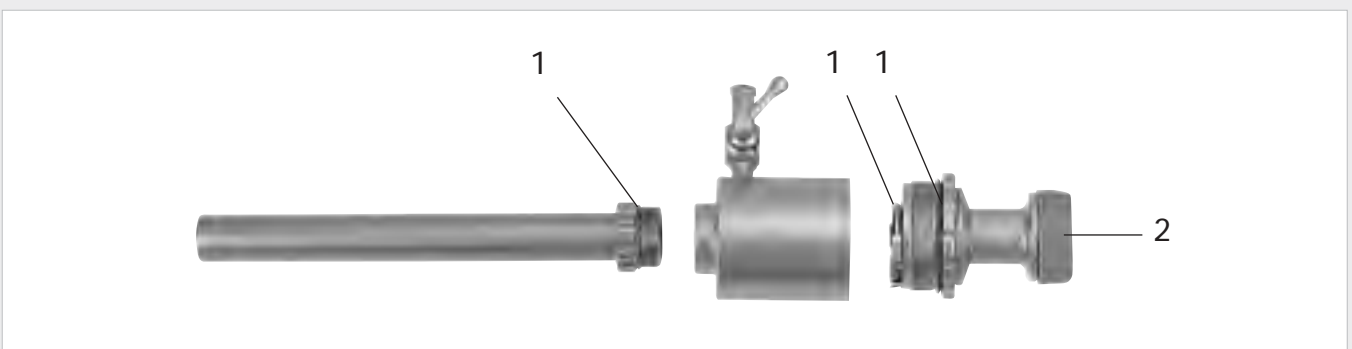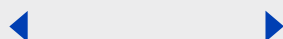
Modular trocar sleeve with silicon valve and exchangeable sleeve. With lateral stop cock for gas insufflation. For standard and safety trocars.

3.0 mm	300-025-003	10.0 mm	300-025-010
5.0 mm	300-025-005	11.0 mm	300-025-011
5.5 mm	300-025-055		

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**Modular Trocar sleeve with trumpet valve** and exchangeable sleeve. With lateral stop cock for gas insufflation. For standard and safety trocars.

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**Modular trocar sleeve with trumpet valve** and exchangeable sleeve, blunt ball-tip obturator and fixable slide-cone with suture holder for **HASSON technique**. With lateral stop cock for gas insufflation.

5.0 mm	300-014-205	12.5 mm	300-014-212
5.5 mm	300-014-255	15.0 mm	300-014-215
10.0 mm	300-014-210	20.0 mm	300-014-220
11.0 mm	300-014-211		

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Fixable delrine slide-cone for open laparoscopy to slide onto all modular trocar sleeves

5.0 mm	300-014-405	12.5 mm	300-014-412
5.5 mm	300-014-455	15.0 mm	300-014-415
10.0 mm	300-014-410	20.0 mm	300-014-420
11.0 mm	300-014-411		

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Modular system for distension in peritoneal surgery using a silicon valve and a reusable obturator in conjunction with a disposable shaft and balloon. This instrument provides rapid access to the peritoneal cavity and maintaining it under direct visualization especially for extraperitoneal hernia repair.

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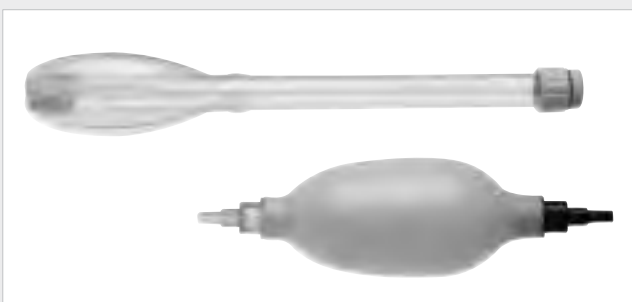
Reusable, blunt obturator 11 mm, for insertion of the structural balloon.

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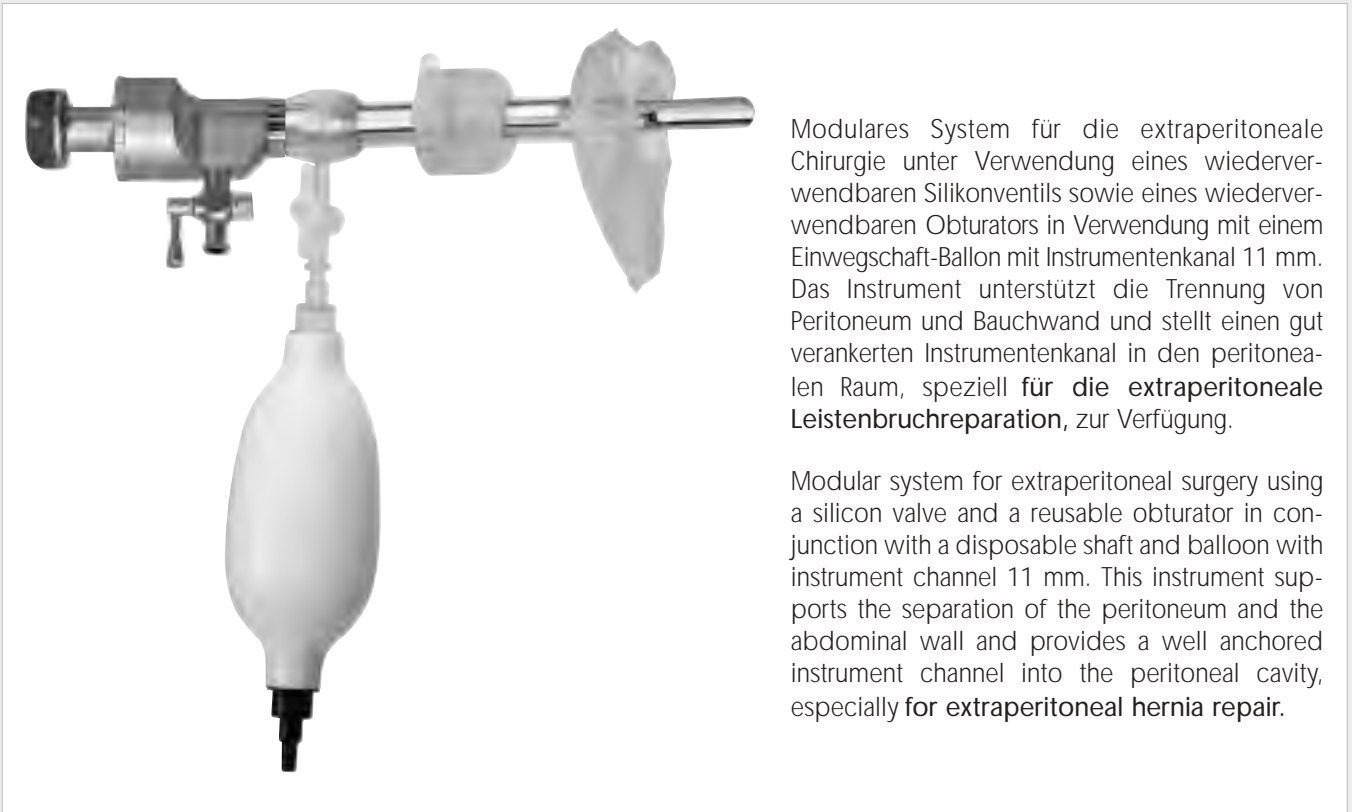
Sterile disposable set 11 mm, with distension balloon for extraperitoneal surgery.  
Contents: Plastic guide tube with round balloon and ball pump.  
(5 units per box)

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Modular system for extraperitoneal surgery using a silicon valve and a reusable obturator in conjunction with a disposable shaft and balloon with instrument channel 11 mm. This instrument supports the separation of the peritoneum and the abdominal wall and provides a well anchored instrument channel into the peritoneal cavity, especially for extraperitoneal hernia repair.

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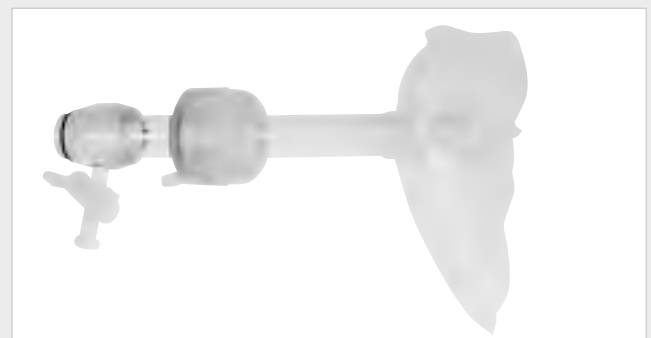
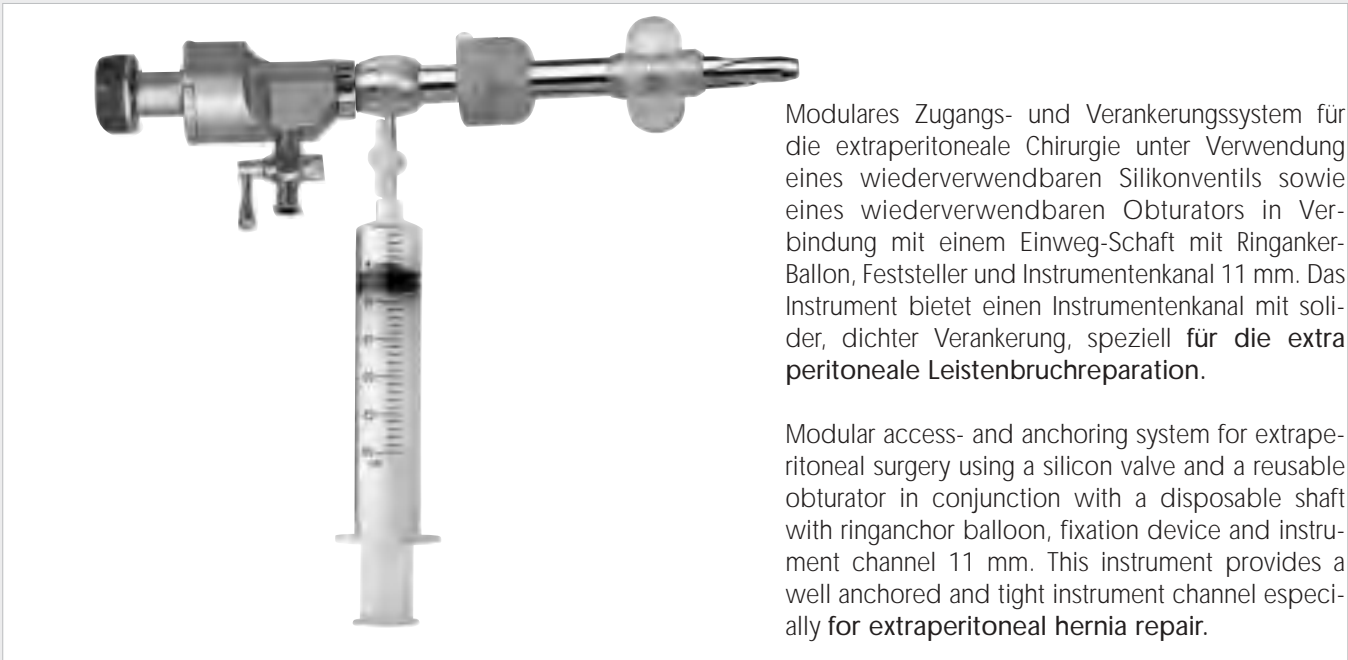
Sterile disposable set 11 mm, with distension balloon for extraperitoneal surgery.  
Contents: Plastic guide tube with balloon and ball pump.  
(5 units per box)

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Modular access- and anchoring system for extraperitoneal surgery using a silicon valve and a reusable obturator in conjunction with a disposable shaft with ringanchor balloon, fixation device and instrument channel 11 mm. This instrument provides a well anchored and tight instrument channel especially for extraperitoneal hernia repair.

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Silicon valve, 11.0 mm  $\varnothing$  for exchangeable trocar sleeves

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Sterile disposable set 11 mm, with distension balloon for extraperitoneal surgery.

Contents: Plastic guide tube with round balloon and ball pump. (5 units per box)

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### RZ-Safety Trocars

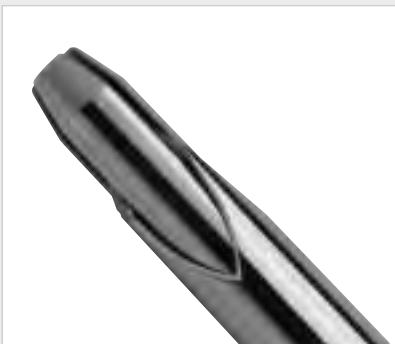
RZ - Safety trocars featuring a self-locking safety mechanism. By rotating the black activation ring the safety mechanism is activated. The obturator slides back and releases the trocar tip while the safety trocar is pushed through the abdomen. Once pushed through the abdominal wall, the obturator bounces back and locks itself, thus permanently protecting the trocar tip.

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RZ-Safety-Trocar, cutting, with automatic, self-locking edge cover.

5.0 mm	300-152-005	11.0 mm	300-152-011
5.5 mm	300-152-055	12.5 mm	300-152-012
7.0 mm	300-152-007	15.0 mm	300-152-015
10.0 mm	300-152-010		

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
Shielded, locked trocar tip to prevent damage to the abdominal cavities.

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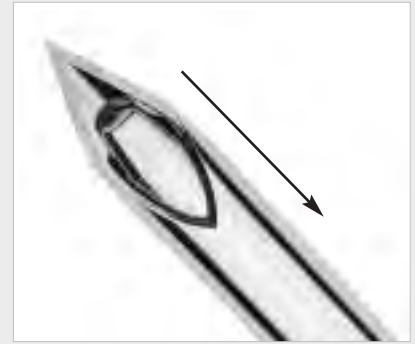Activation of the safety mechanism.

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Un-shielded trocar tip while being pushed through the abdominal wall.

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
Reducer sleeve and appendix extractor for use  
with 5 mm / 10 mm instruments.

Instrument Size	Reducer Sleeve	Appendix Extractor
ø 10.0 → 5.0 mm	300-012-106	300-012-105
ø 11.0 → 5.5 mm	300-012-116	300-012-115
ø 12.5 → 5.5 mm	300-012-121	300-012-120
ø 15.0 → 5.5 mm	300-012-557	300-012-555
ø 20.0 → 5.5 mm	300-012-558	300-012-556
ø 15.0 → 10.0 mm	300-012-666	300-012-665
ø 20.0 → 10.0 mm	300-012-202	300-012-201

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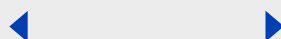Reusable **modular trocar sleeves without valves** for gasless laparoscopy. Exchangeable guide tube for standard-and safety-trocars.

∅ 5.0 mm	300-009-005
∅ 5.5 mm	300-009-055
∅ 10.0 mm	300-009-010
∅ 11.0 mm	300-009-011
∅ 12.5 mm	300-009-025
∅ 15.0 mm	300-009-015

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Tissue retractors, S-shaped, set of two pieces to expose anatomical layers.

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Endo-dilator

To extend the incision and for an easy extraction of organs in laparoscopic surgery - e.g. removal of a large gallbladder. Prior to a cholecystectomy the extractor is slid over the trocar sleeve in the umbilical area until the blades get visible in the abdominal cavity. Then the isolated gallbladder is pulled into the trocar sleeve. The Endodilator is spread as wide as possible and at the same time the trocar with gallbladder is pulled out through the dilator.

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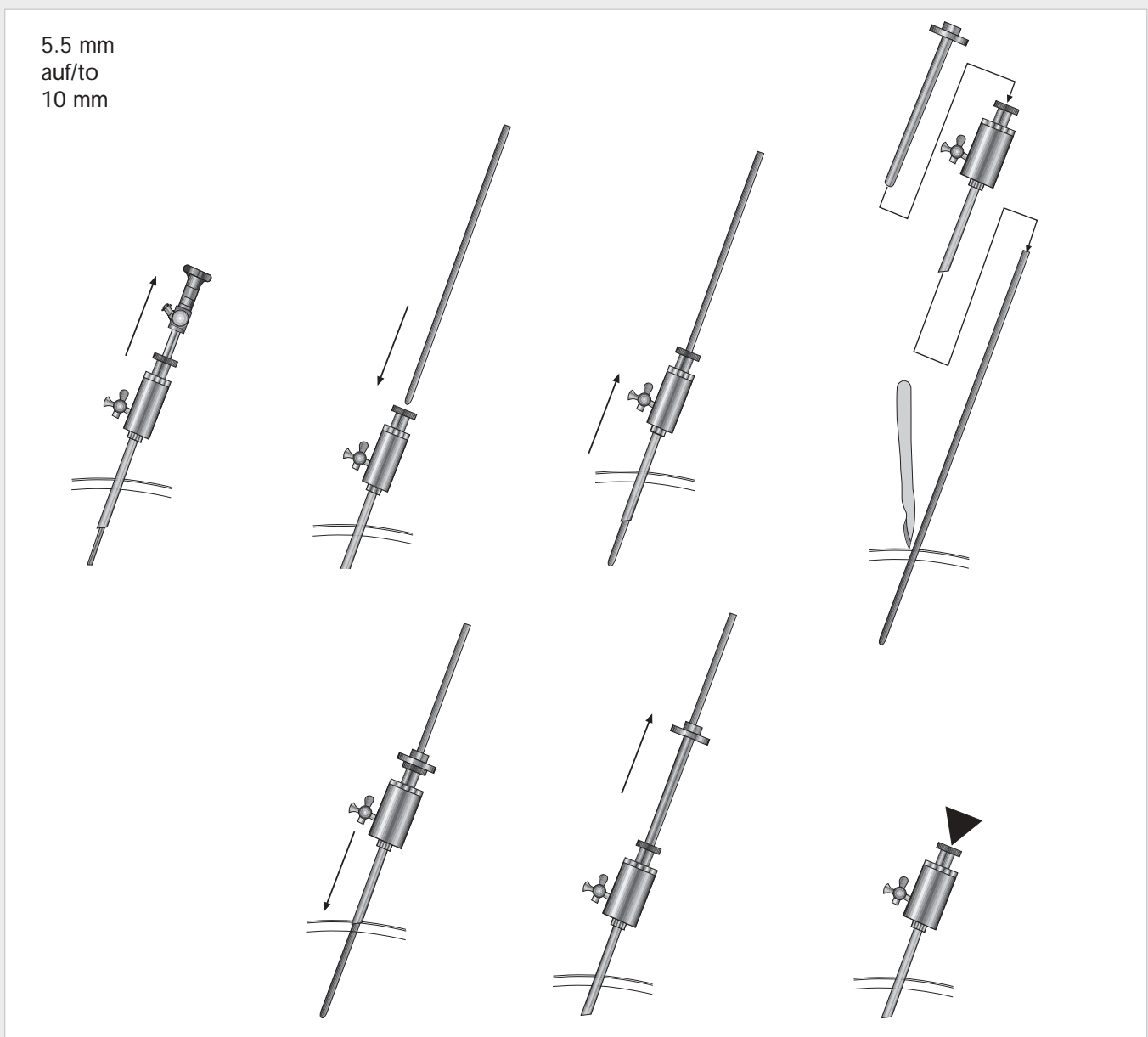
By using the Dilation Instrument-Set an exchange of a 5.5 or 11 mm trocar to a larger size trocar is possible without any further incision of the peritoneum.

RZ Dilatations-Set / Dilation Set

für	5.5 →	11.0 mm	300-013-511
für	5.5 →	10.0 mm	300-013-510
für	11.0 →	15.0 mm	300-013-115
für	5.5 →	12.5 mm	300-013-512
für	11.0 →	20.0 mm	300-013-522

Veress Insufflationsnadel, 2 mm  
Veress Insufflation needle, 2 mm

55 mm	300-195-055
70 mm	300-195-070*
80 mm	300-195-080
100 mm	300-195-100*
120 mm	300-195-120*
140 mm	300-195-140
150 mm	300-195-150*
170 mm	300-195-170
200 mm	300-195-200

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Adaptable cutting introducer cannula, sterile,  
2 mm (10 pcs. per box )

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
Complete system, consisting of: modular insuffla-  
tion needle and cutting introducer cannula

Länge length	Adaptierbare Außenkanüle Adaptable outer cannula in Packungen zu 10 St., in packages of 10 pcs.	Modulare Insufflationsnadel modular Insufflation needle		kompletter Satz complete set
		unsteril / non sterile	steril / sterile	
70 mm	300-198-070	300-199-070	300-197-070	300-196-070
100 mm	300-198-100	300-199-100	300-197-100	300-196-100
120 mm	300-198-120	300-199-120	300-197-120	300-196-120
150 mm	300-198-150	300-199-150	300-197-150	300-196-150
170 mm	300-198-170	300-199-170	300-197-170	300-196-170
200 mm	300-198-200	300-199-200	300-197-200	300-196-200

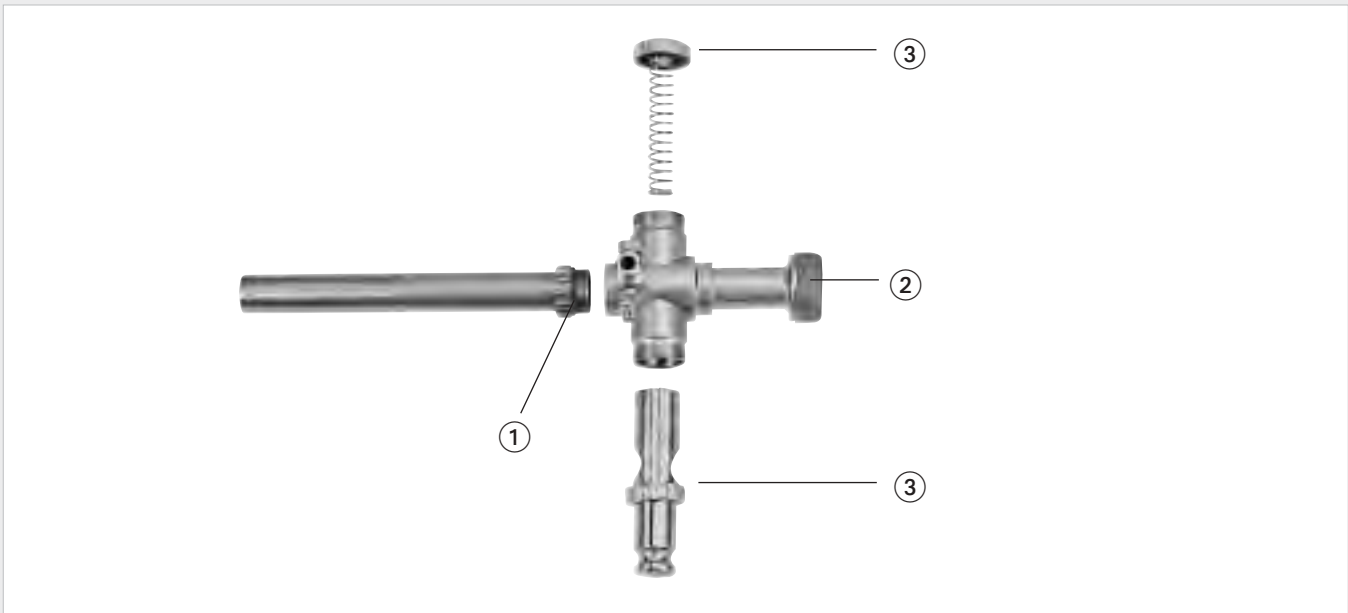	1	3	2
Trocarhülsen Trocar sleeves	Dichtungsring gaskets	komplettes Ventilset complete valve set	Dichtungskappe, blau sealing cap, blue
	in Packungen zu 10 St. in packages of 10 pcs.	inkl. Ventilkappe, Feder, Bolzen, oberer Ventilkappe incl. Valve cap, spring valve, pin, upper valve cap	in Packungen zu 10 St. in packages of 10 pcs.

5.0 mm	300-021-905	300-022-905	300-023-905
5.5 mm	300-021-955	300-022-955	300-023-955
7.0 mm	300-021-907	300-022-907	300-023-907
10.0 mm	300-021-910	300-022-910	300-023-910
11.0 mm	300-021-911	300-022-911	300-023-911
12.5 mm	300-021-912	300-022-912	300-023-912
15.0 mm	300-021-915	300-022-915	300-023-915

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RZ-Medizintechnik GmbH manufactures a wide range of multifunctioning suction- and irrigation systems. These can be used for various endoscopic surgical techniques. To enable our products to function with other systems, we have attached various connectors.

An information chart is provided showing the advantage of the individual systems which will facilitate their operation.

The main features of our suction- and irrigation systems are the delivery of a precise amount of fluid via the irrigating systems, allied to simple and comfortable usage.

Our instruments are equipped with a size M8 x 1 mm thread, which enables the attachment of eight various accessories to all three available piston handles.

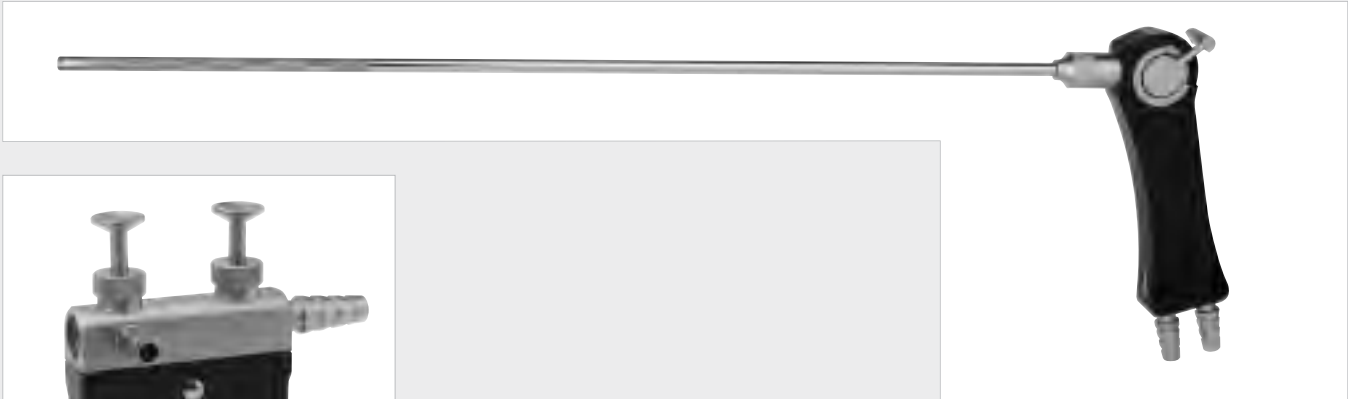
These handles are ergonomically designed, which facilitates continuous operation without undue fatigue.

This system has the advantage of a rotating adapter, which allows the use of rotating attachments. This presents a considerable advantage when using the RZ optical irrigation System. Furthermore this instrument is designed with a Male Luer-Lock adapter for additional security during operation; preventing unintentional unscrewing during the rotation of the instruments.

To increase the working range of our suction instruments, we have attached a monopolar poste for the connection to H.F.; this allows the connection of various suction electrodes - simultaneous coagulation, suction and irrigation can then be achieved.

Our well-designed system, incorporating a rotating adaptor, permits the operator to use coagulation without the necessity to rotate the instrument.

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Suction/Irrigation handle with two trumpet valves and detachable ergonomic handle plate; for adaptable tubes with connector M8 x 1 mm.  
One universal hose connector; one lateral, female Luer Lock connector.

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Suction/Irrigation pistol handle with maintenance-free 2-way valve for adaptable tubes with connector M8 x 1 mm, two universal hose connectors.

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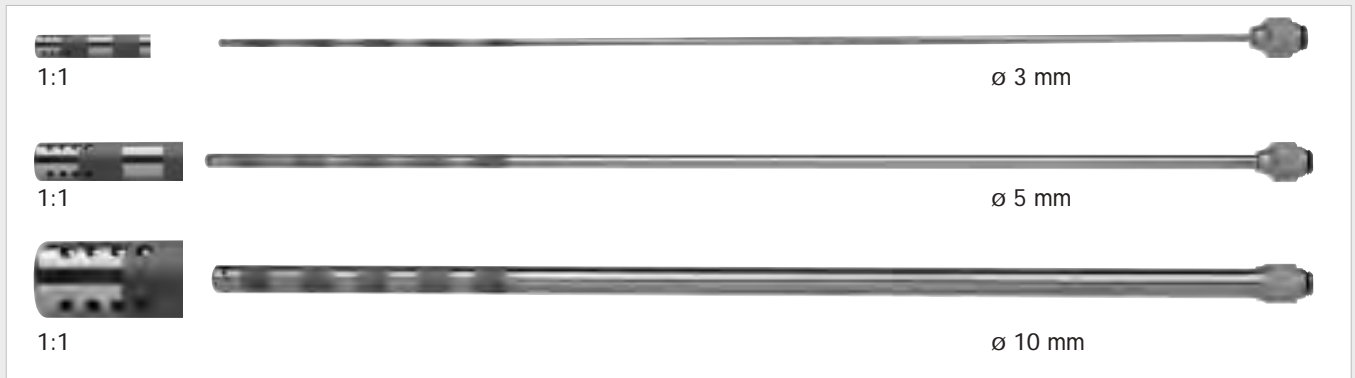
Suction/Irrigation handle with trumpet valve for adaptable tubes with connector M8 x 1 mm, one universal hose connector.

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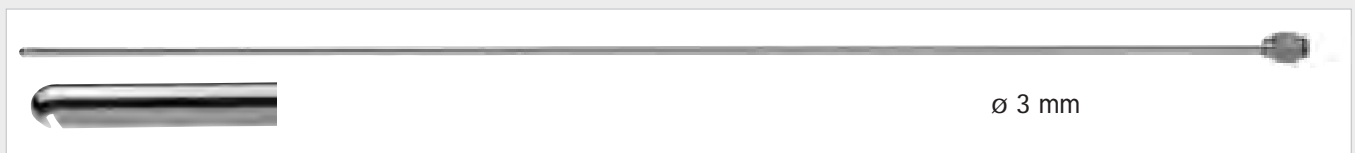
Adaptable suction/irrigation tube with connector M8 x 1 mm and 4 lateral holes at the distal end. Can be used with all handles 300-167-100, 300-165-100 and 300-166-100

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Adaptable suction/irrigation tube with connector M8 x 1 mm and 4 x 4 lateral holes at the distal end. Can be used with all handles 300-167-100, 300-165-100 and 300-166-100

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Adaptable optics flushing tube with angled flush opening and connector M8 x 1 mm. Can be used with all handles 300-165-100, 300-166-100 and 300-167-100

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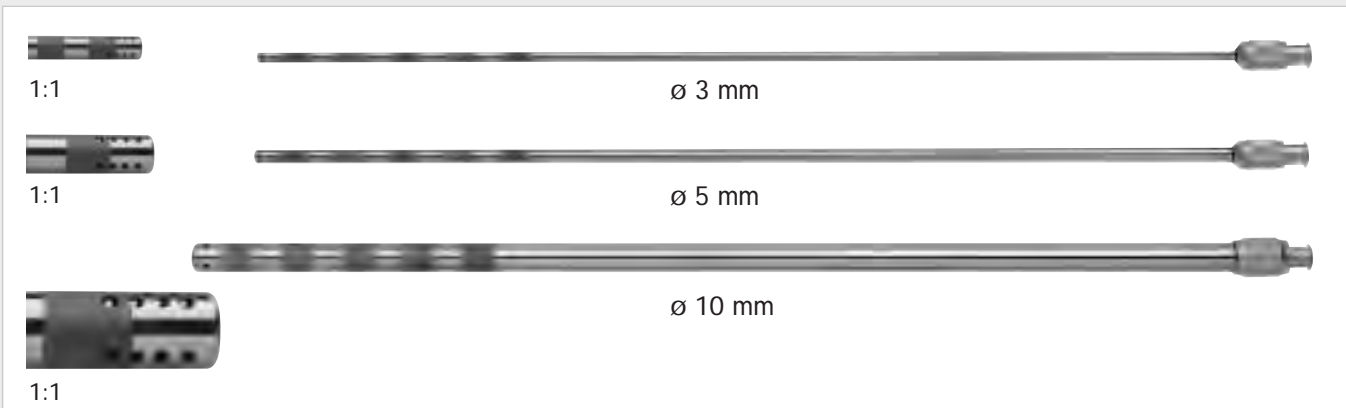
Adaptable suction/irrigation tube with puncture/injection needle (1 mm) and connector M8 x 1 mm. Can be used with all handles 300-165-100, 300-166-100 and 300-167-100

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Maintenance-free 2-way valve with rotating adaptor, large Luer-Lock-male and 2 Luer-Lock-female connectors with hose-olives.

	Größe/Size
300-192-003	ø 3 mm
300-192-005	ø 5 mm
300-192-010	ø 10 mm

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Adaptable suction/irrigation tube with large Luer-Lock Connector female and 4 lateral holes at the distal end. **Can be used with the handle 300-192-000.**

300-193-003	ø 3 mm
300-193-005	ø 5 mm
300-193-010	ø 10 mm

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Adaptable suction/irrigation tube with Luer-Lock Connector and 4 x 4 lateral holes at the distal end. **Can be used with the handle 300-192-000.**

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Adaptable optics flushing tube with angled flush opening and large Luer-Lock Connector female. **Can be used with the handle 300-192-000.**

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Adaptable suction/irrigation tube with punction/injection needle ø 1 mm and large Luer-Lock Connector female. **Can be used with the handle 300-192-000.**

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Irrigation- and suction cannula set with two **large** trumpet valves and interchangeable suction tubes 5 mm and 10 mm, 33 cm

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Valve body with two large trumpet valves, with tube connector

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Accessories for gynaecological surgeries

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Suction tube with puncture needle, 5 mm, 30 cm

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Suction tube with puncture needle, 5 mm, 45 cm

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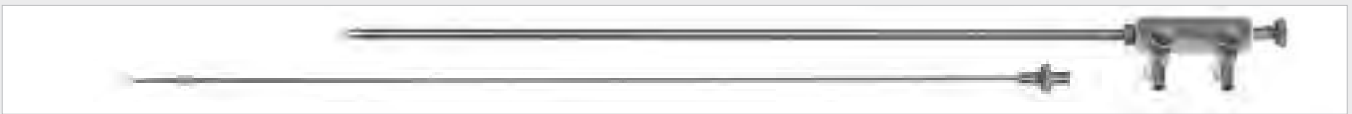
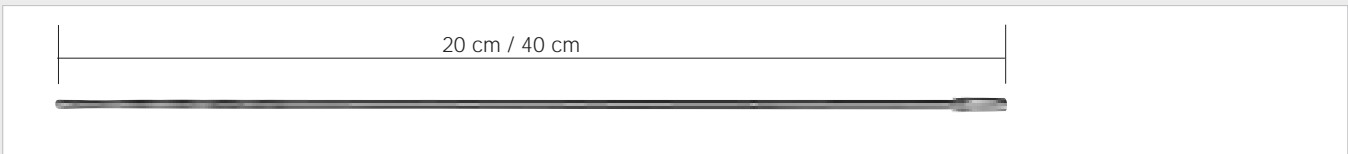
300-103-620 ø 3.5 mm  
Ovariennadel, Luer-Lock-Ansatz / Nadel ø 1,2 mm, 20 cm

Ovary needle, Luer-Lock-connection  
Needle ø 1.2 mm, 20 cm

300-125-005 ø 5 mm  
Ovariennadel, Luer-Lock-Ansatz / Nadel ø 1,2 mm, 33 cm

Ovary needle, Luer-Lock-connection  
Needle ø 1.2 mm, 33 cm


300-103-335 ø 5 mm, 20 cm  
Tastsonde graduiert  
300-193-335 ø 5 mm, 40 cm  
Tastsonde graduiert

Palpation probe graduated  
Palpation probe graduated

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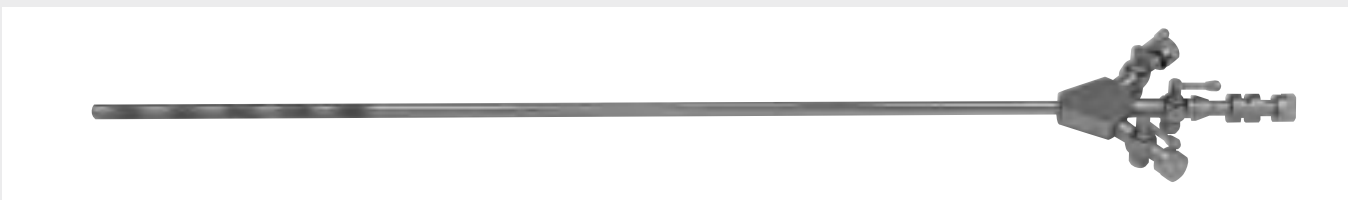Suction-irrigation tube with two lateral trumpet valves,  
graduated, with laser channel.

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Suction-irrigation tube with three maintenance-free stop  
cocks and laser channel, graduated.

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The multifunctional-Suction- and Irrigation cannula can be perfectly used for the universal application of monopolar electrodes as well as of bipolar electrodes in laparoscopy.

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Multifunctional Irrigation- and Suction cannula, 10 mm, with two lateral trumpet valves for irrigation and one 5 mm trumpet valve for instruments.

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Bipolar coagulation electrodes, 5 mm, working length: 380 mm

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Scalpel blades for 10 mm multifunctional suction- and irrigation system

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Scalpel blade holder, ø 5 mm, working length: 380 mm for ø 10 mm suction- and irrigation system.

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Suction- and Irrigation cannula 5 mm, with two lateral trumpet valves and Luer-Lock connection

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Suction and irrigation cannula with laser guide

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Bipolar coagulation electrode for 5 mm multifunctional suction- and irrigation cannula

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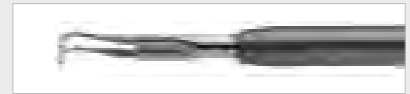
Scalpel blades for ø 5 mm multifunctional suction- and irrigation system

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Scalpel blade holder for ø 5 mm multifunctional suction- and irrigation system, ø 3 mm, 400 mm

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Note:

The instrument with working length 44 cm is compatible to the multifunction suction-/irrigation tube ø 10 mm, 300-265-100

	Typ A 34 cm	Typ B 34 cm	Typ B 44 cm	HF-Elektroden HF-Electrodes
J-Haken-Elektrode / Hook-Electrode thin einzel / single 5 St. / pcs. steril / sterile	300-163-101	300-163-200 300-163-500	300-163-300 300-163-600	
J-Haken-Elektrode / Hook-Electrode thick einzel / single 5 St. / pcs. steril / sterile	300-163-111	300-163-210 300-163-510	300-163-310 300-163-610	
L-Haken-Elektrode / Right-Angle-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-121	300-163-220 300-163-520	300-163-320 300-163-620	
Spatel-Elektrode / Spatula-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-131	300-163-230 300-163-530	300-163-330 300-163-630	
Knopf-Elektrode / Ballpoint-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-141	300-163-240 300-163-540	300-163-340 300-163-640	
Messer-Elektrode / Knife-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-151	300-163-250 300-163-550	300-163-350 300-163-650	
45°-Elektrode / 45°-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-161	300-163-260 300-163-560	300-163-360 300-163-660	
Nadel-Elektrode / Needle-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-171	300-163-270 300-163-570	300-163-370 300-163-670	
Konus-Elektrode / Cone-Electrode einzel / single 5 St. / pcs. steril / sterile	300-163-181	300-163-280 300-163-580	300-163-380 300-163-680	

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
Systems for monopolar electro-surgery as well as for irrigation and aspiration of rinsing liquids in the laparoscopic operating area. All systems and handles can be connected and disconnected without any tools. Each handle can be used with any adaptable monopolar HF-electrode with suction/irrigation channel of series 300-163-... i. e. all handles and HF-electrodes of this series are completely interchangeable.

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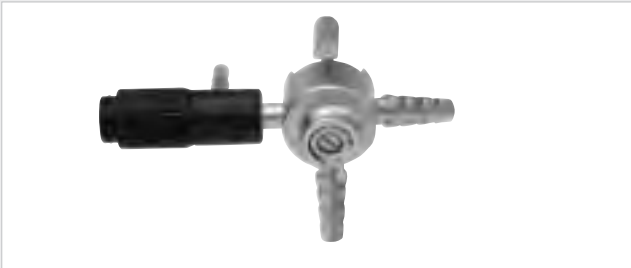
Trumpet-valve handle for adaptable HF-electrodes with suction/irrigation channel. With rotating adaptor and lateral HF-connector, one universal hose connector.

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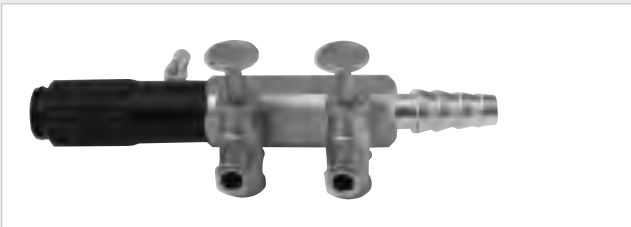Maintenance-free two-way valve for adaptable HF-electrodes with suction/irrigation channel. With rotating adaptor and lateral HF-connector, two universal hose connectors.

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Handle with two lateral trumpet valves for adaptable HF-electrodes with suction/irrigation channel. With rotating adaptor and lateral HF-connector, one universal hose connector and two female Luer-Lock Valve connectors.

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Handle with two trumpet valves and detachable ergonomic handle plate; for adaptable HF-electrodes with suction/irrigation channel. With rotating adaptor and lateral HF-connector. One universal hose connector and one female Luer-Lock valve connector.

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Pistol handle with dual trumpet valve for adaptable HF-electrodes with suction/irrigation channel. With rotating adaptor and lateral HF-connector, two universal hose connectors.

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All modular HF-Electrodes with suction/irrigation channel can be adapted to all modular valve handles of series 300-165-000, 300-166-000, 300-167-000, 300-169-000, 300-170-000. The connection is done with double security by means of a female Luer-Lock connector which is additionally tightened through an O-ring system. The tube insulation consists of a very resistant plastic tube, which is drawn up onto a stainless steel core tube.

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Modular Multifunction Suction-/Irrigation System with pistol handle, maintenance-free 2-way valve, many different adaptable tubes and sealable instrument channel  $\varnothing$  5 mm for irrigation and aspiration of rinsing liquid in the laparoscopic operating area. The instrument channel automatically opens when instruments are inserted and closes after they have been removed. Alternatively the closure of the instrument channel can be opened manually by pressing a pushbutton for example to remove a hook-type instrument or a biopsy. The system can be disassembled/assembled for cleaning without any additional tools needed.

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Suction-/Irrigation pistol handle with maintenance-free 2-way valve and instrument channel  $\varnothing$  5 mm. With automatic and manual operating closure and two universal hose connectors. All tubes of series 300-166-... with a M8 x 1 mm connector are adaptable.

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Adaptable metal suction-/irrigation tube, outside  $\varnothing$  10 mm, with connector M8 x 1 mm, for use with instruments  $\varnothing$  5 mm / 330 mm working length. inside  $\varnothing$  at distal end:  $\varnothing$  6.0 mm, working length: 230 mm.

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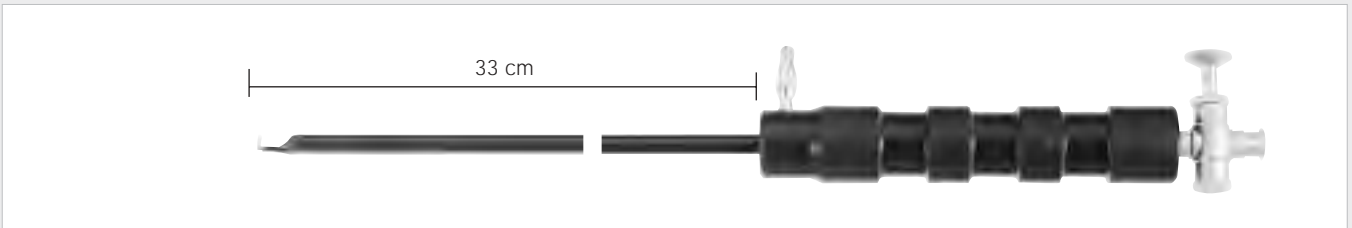
Adaptable plastic suction-/irrigation tube, outside  $\varnothing$  10 mm, with connector M8 x 1 mm, for use with instruments  $\varnothing$  5 mm / 330 mm working length. inside  $\varnothing$  at distal end:  $\varnothing$  6.0 mm, working length: 230 mm.

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
**Type A**  
Electrodes ø 5 mm / 33 cm with HF-connector monopolar

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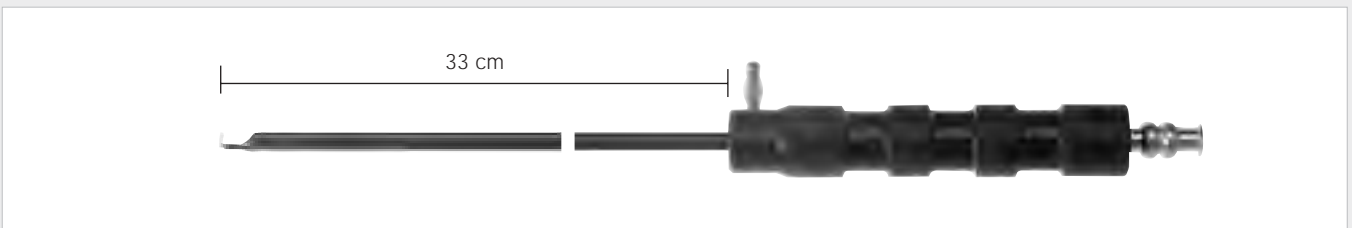**Type B**  
Suction electrodes ø 5 mm / 33 cm with HF-connector, monopolar, with trumpet valve

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**Type C**  
Suction electrodes ø 5 mm / 33 cm with HF-connector, monopolar, with Luer-Lock

	Typ/Type A	Typ/Type B	Typ/Type C
Saugelektrode Suction electrode	-----	300-100-148	300-100-187
Spatelektrode Spatula electrode	300-100-131	300-100-147	300-100-186
J-Haken-Elektrode J-Hook-Electrode	300-100-132	300-100-140	300-100-190
L-Haken-Elektrode L-Hook-Electrode	300-100-133	300-100-141	300-100-185
Knopf-Elektrode Ball-Electrode	300-100-134	300-100-146	300-100-188

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## For Application of HF-Auxiliary Instruments ...with unipolar high-frequency connection\*

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In instruments with unipolar connection, the high frequency current flows through the small area electrode into the tissue with a high current density. As of a certain current density, coagulation will occur. The current returns over large cross sections of tissue - without coagulation effect - to the large area patient plate and finally back to the generator via the cable connection.

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1. The max. output power of the unit has to be matched to the application. Units having excessive output power cannot be adjusted finely enough in the low power range. In gastroenterology, gynecology and rhinoloaryngology the unipolar output power should not exceed 120 watts, units with incorporated coagulation current source for blended current 170 watts. For the TUR units with 300-400 watts with matched output circuit for underwater cutting are required. The neutral electrode should be grounded for the high frequency Current (no floating output).

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2. Neutral electrode: do not use any bent or misshapen plates - fasten to the patient's upper thigh. Ensure large-area contact with the clean, hairless skin. It is beneficial to wipe the skin in advance with a little water. The patient should not come into contact with other metal items (e.g. operating table, retainers). The same applies to damp cloths, etc., especially damp underlays.

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The neutral electrode should be attached near the operating area. The figure shows the optimal attachment places and their operating areas.

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3. Connecting cables leading to the neutral and active electrodes must be in perfect condition. The cables must be checked for current passage and for undamaged isolation and connecting plugs. With cables which are disinfected by immersing them into a solution, the connectors should be wiped dry on inner and outer side using a sterilized cloth or swab. Connecting cables supplied by RZ-Medizintechnik GmbH company must be used exclusively for these high-frequency connections.

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4. If the patient is simultaneously connected to high-frequency unit and a monitoring unit (e.g. EGG), special attention must be paid to the following. If both units are operated with earthed neutral electrodes, the neutral EGG cable must also be connected to the neutral surgery electrode. The high-frequency units must not be used in the close vicinity of the EGG electrode (min. distance 150 mm).

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5. The insulation of the instruments and electrodes (even with only small pressure marks on the insulation, caused by forceps) may no longer be used. Only touch plastic coated handles with rubber gloves.

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6. The power should be set on the unit only as high as required for the operation. Guideline values should be obtained extracorporeally. If the required coagulation or cutting effect is not achieved at the usual setting, the power may be increased, provided that the neutral electrode is firmly in position, all lines and plug connections are in perfect condition the instrument insulation has been checked, the metallic parts of the instruments acting as electrodes and the electrode are perfectly clean.

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**Multiple Puncture Approach**

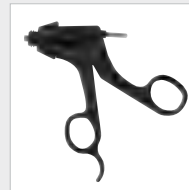
Operating instruments for use with 5.0 mm and 5.5 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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
**Single Puncture Approach**

Operating instruments for use with 5.0 mm and 5.5 mm trocar, 45 cm working length of instruments. For the use with all outer tubes and handles.

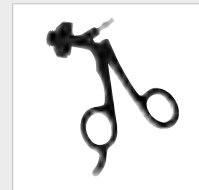Komplett-Instrument  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

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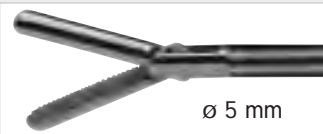
Kelly Dissecting and Grasping Forceps, double action jaws, very delicate

	21 cm	300-402-002	300-412-002	300-422-002
	33 cm	300-400-002	300-410-002	300-420-002
	45 cm	300-401-002	300-411-002	300-421-002

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Dissecting and Grasping Forceps, long jaws, double action jaws, delicate pattern, standard serrations

	21 cm	300-402-001	300-412-001	300-422-001
	33 cm	300-400-001	300-410-001	300-420-001
	45 cm	300-401-001	300-411-001	300-421-001

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Dissecting and Grasping Forceps, double action jaws, heavy pattern, standard serrations

	21 cm	300-402-021	300-412-021	300-422-021
	33 cm	300-400-021	300-410-021	300-420-021
	45 cm	300-401-021	300-411-021	300-421-021

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
Delphin Dissecting and Grasping Forceps, double action jaws, short jaws, standard serrations

	21 cm	300-402-024	300-412-024	300-422-024
	33 cm	300-400-024	300-410-024	300-420-024
	45 cm	300-401-024	300-411-024	300-421-024

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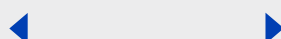Duckbill Dissecting and Grasping Forceps, double action jaws, long jaws, standard serrations

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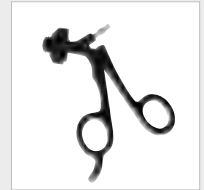
Multiple Puncture Approach  
 Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm and 11.0 mm trocars, 33 cm working length of instruments. For the use with all outer tubes and handles.

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
Single Puncture Approach  
 Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm und 11.0 mm trocars, 45 cm working length of instruments. For the use with all outer tubes and handles.


Komplett-Instrument  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

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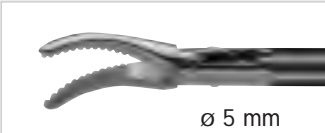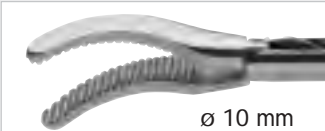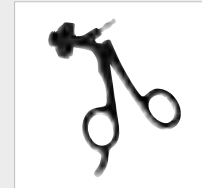
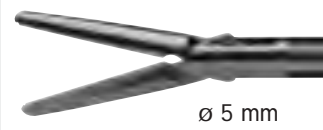
**Multiple Puncture Approach**

Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm and 11.0 mm trocars, 33 cm working length of instruments. For the use with all outer tubes and handles.

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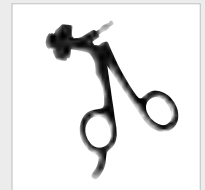
**Single Puncture Approach**

Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm und 11.0 mm trocars, 45 cm working length of instruments. For the use with all outer tubes and handles.


Komplett-Instrument  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

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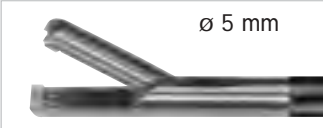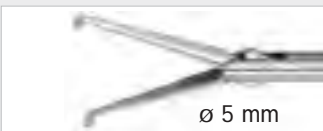
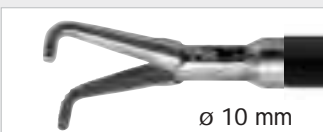
**Multiple Puncture Approach**

Operating instruments for use with 5.0 mm, 5.5 mm and 10.0 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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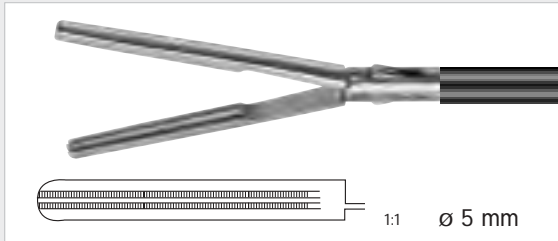
**Multiple Puncture Approach**

Operating instruments for use with 5.5 mm, 10.0 mm and 11.0 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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
**Single Puncture Approach**

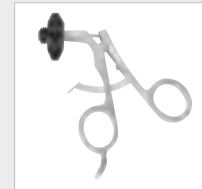
Operating instruments for use with 5.5 mm, 10.0 mm and 11.0 mm trocar, 45 cm working length of instruments. For the use with all outer tubes and handles


Komplett-Instrument  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

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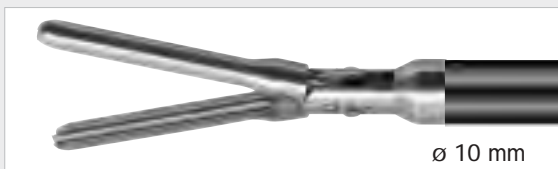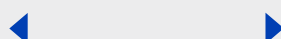
**Multiple Puncture Approach**

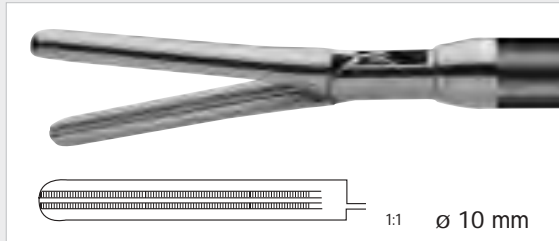Operating instruments for use with 10.0 mm and 11.0 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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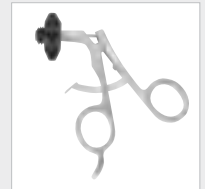
**Single Puncture Approach**

Operating instruments for use with 10.0 mm and 11.0 mm trocar, 45 cm working length of instruments. For the use with all outer tubes and handles

**Komplett-Instrument**  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

**Komplett-Instrument**  
 Complete instrument  
 Metallhandgriff  
 Metal handle

Glassman, 10 mm Klemmzange, doppelbeweglich, atraumatische Zahnung	Länge/length	Einsatz/insert	30 mm Maullänge/jaws	
	21 cm	300-402-272	300-412-272	300-422-272
	33 cm	300-400-272	300-410-272	300-420-272
	45 cm	300-401-272	300-411-272	300-421-272
Glassman, 10 mm Forceps, double action jaws, atraumatic serrations			40 mm Maullänge/jaws	
	21 cm	300-402-274	300-412-274	300-422-274
	33 cm	300-400-274	300-410-274	300-420-274
	45 cm	300-401-274	300-411-274	300-421-274
			60 mm Maullänge/jaws	
	21 cm	300-402-276	300-412-276	300-422-276
	33 cm	300-400-276	300-410-276	300-420-276
	45 cm	300-401-276	300-411-276	300-421-276

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**Multiple Puncture Approach**

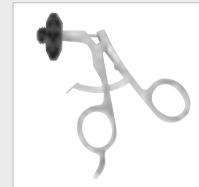
Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm, 10.5 mm and 11.0 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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**Single Puncture Approach**

Operating instruments for use with 5.0 mm, 5.5 mm, 10.0 mm, 10.5 mm and 11.0 mm trocar, 45 cm working length of instruments. For the use with all outer tubes and handles

Komplett-Instrument  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

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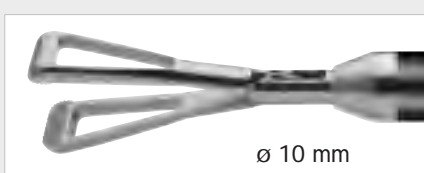
**Multiple Puncture Approach**

Operating instruments for use with 10.0 mm, 10.5 mm and 11.0 mm trocar, 33 cm working length of instruments. For the use with all outer tubes and handles.

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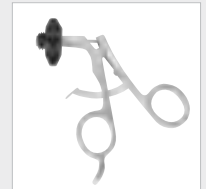
**Single Puncture Approach**

Operating instruments for use with 10.0 mm, 10.5 mm and 11.0 mm trocar, 45 cm working length of instruments. For the use with all outer tubes and handles.

**Komplett-Instrument**  
 Complete instrument  
 Kunststoffhandgriff  
 Carbon Fibre handle

**Komplett-Instrument**  
 Complete instrument  
 Metallhandgriff  
 Metal handle

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### Instructions for Dismantling

Secure the instrument shaft against rotation, by closing the handle completely.

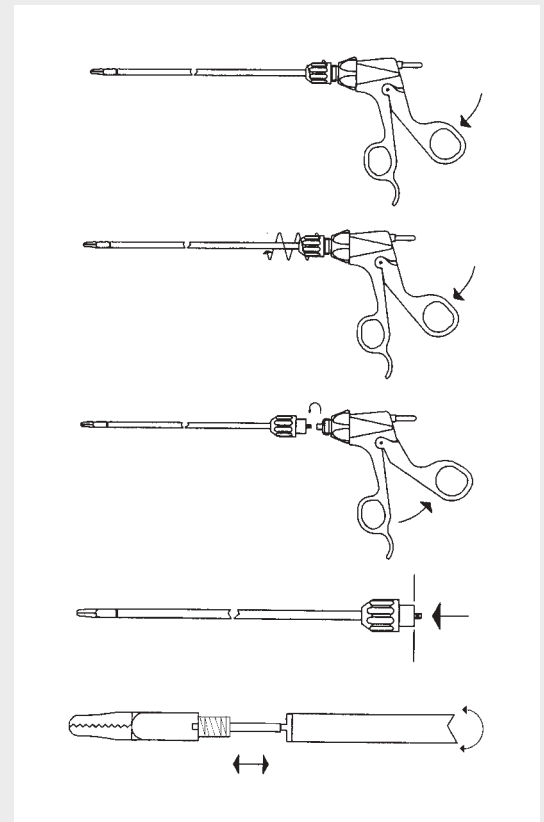
Unscrew the large black collar nut completely by turning in an anti-clockwise direction.

Now open the handle completely ; this will enable you to release the guide tube from the handle by closing the jaws of the working end of the instrument completely and adjusting the white mark on the rotating ring handle to 12 o'clock.

You are now able to unscrew the insert out the guide tube.

The instrument is now completely dismantled into three parts: Handle - Guide Tube - Insert.

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### Instructions for Re-Assembly

Ensure the handle is completely open.

Ensure that the jaws of the working end of the instrument are completely closed.

Place the insert of the instrument into the guide tube and screw together.

Check that the white mark on the rotation ring (of the ring handle) remains at 12 o'clock.

Realign the lug rod in line with the white mark on the rotation ring.

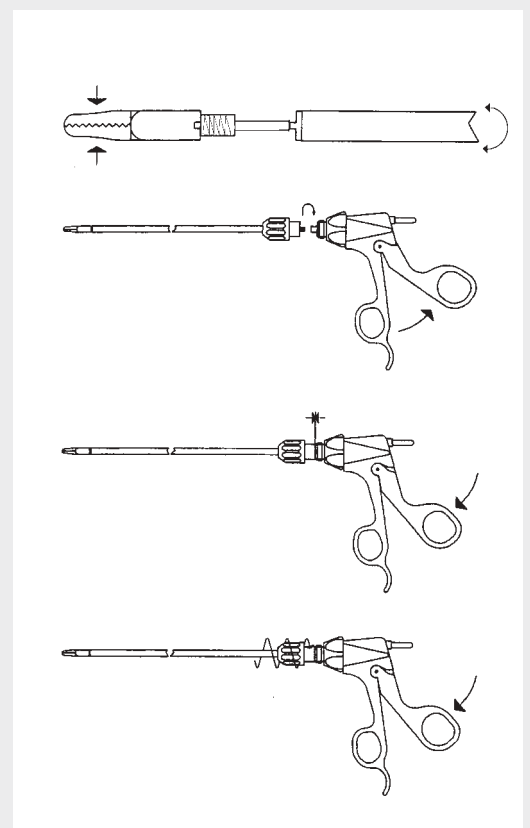
Now close the handle completely.

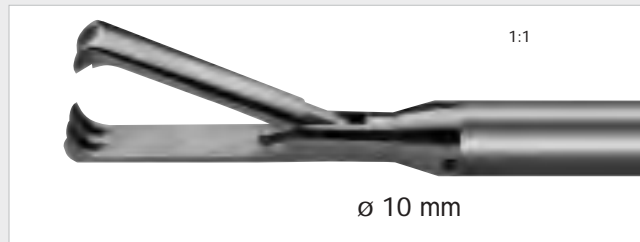
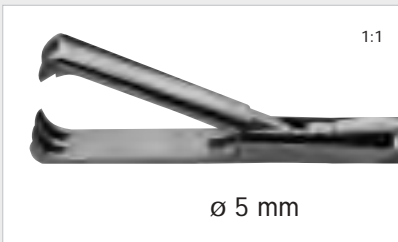
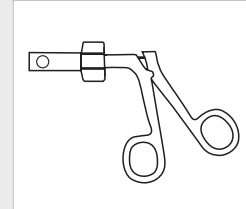
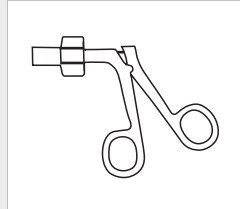
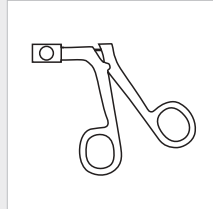
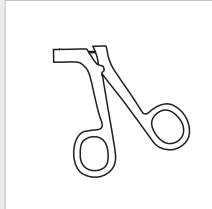
The instrument is now reassembled and ready to be made secure.

This is achieved by connecting the large black collar screw to the male screw thread and screwing tightly together.

The instrument should now be in working order. Please test to confirm this.

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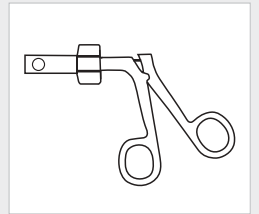
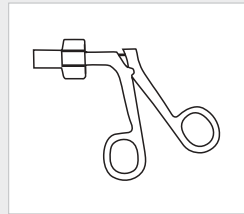
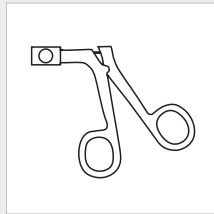
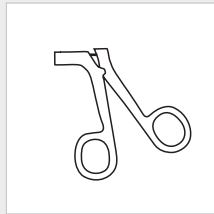
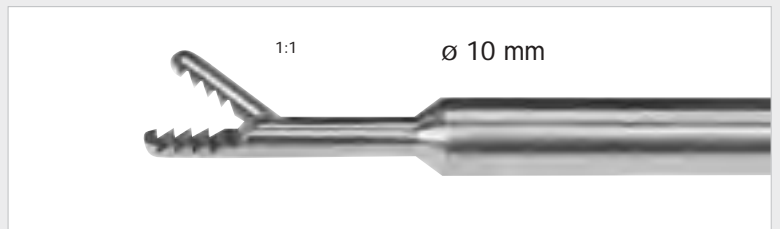
Grasping and Extracting Forceps, serrated jaws,  
 handle with spring

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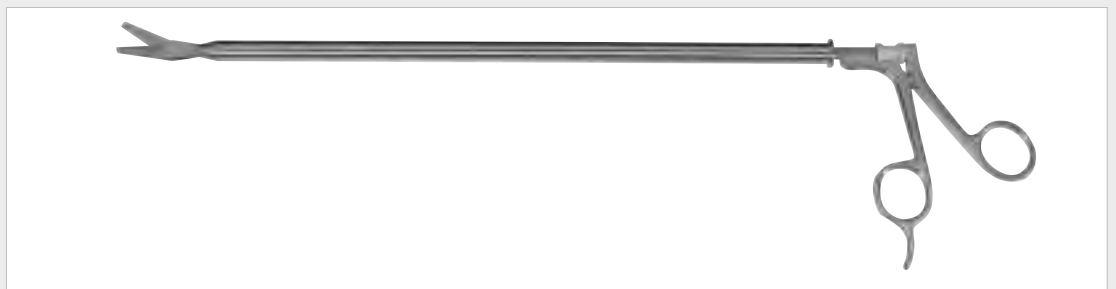
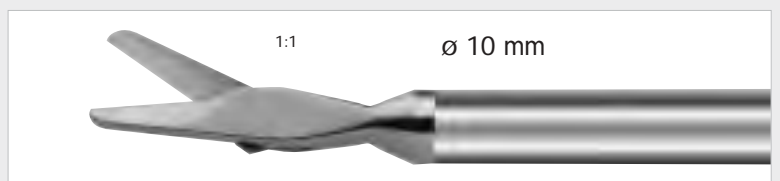
Endoscopic Scissors Mayo model, one blade  
 serrated, straight

300-100-760 Cholangiographie Fixierklemme mit Kanal 6 Ch.,  
5 mm, 33 cm lang, mit Sperre  
Cholangiography-fixation clamp with channel 6 Ch./Fr.,  
5 mm, 33 cm length, with ratchet

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Clip applying forceps, ø 10 mm, length 33 cm,  
360° rotatable, with LL-rinsing channel, for TITAN-Clips

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
300-801-001	Ethicon - small
300-801-002	Ethicon - medium
300-801-004	Ethicon - medium-large
300-801-006	Ethicon - large
300-801-007	Pilling/Weck - small
300-801-008	Pilling/Weck - medium
300-801-010	Pilling/Weck - medium-large
300-801-011	Horizon - small
300-801-012	Horizon - medium
300-801-014	Horizon - medium-large

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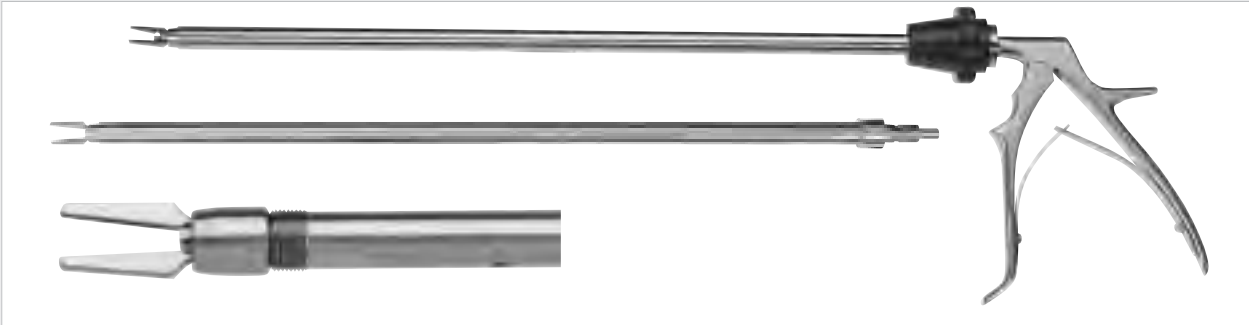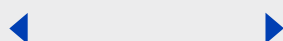
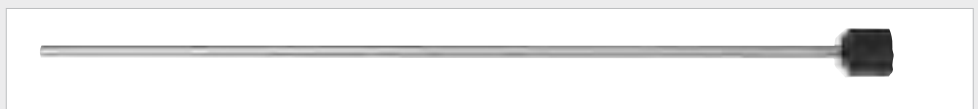
300-811-105  
 Griff alleine  
 Handle, only

300-400-775 Endoskopische Bulldog Klemmen-  
Anlegezangen, zerlegbar

Endoscopic Bulldog Clamp Applicator, detachable

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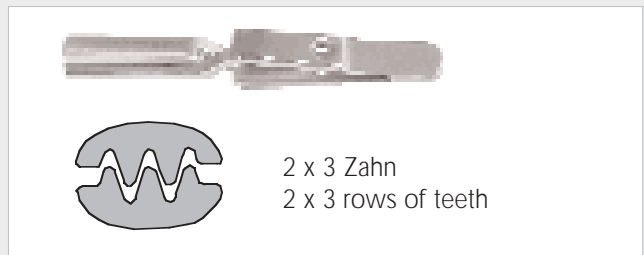
Endoscopic Bulldog Clamps with DeBakey Serration

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Advantages:

- Secure holding of occluded tissue structure
- Occlusion of vessel and other tissue without an additional port
- Atraumatic cross clamping of the colon

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
Endoscopic Vascular Bulldog Clamp  
17 mm jaws with DeBakey serration

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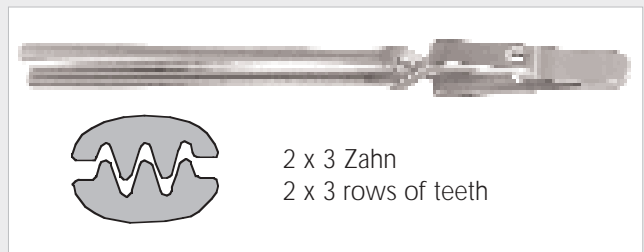Endoscopic Vascular Bulldog Clamp  
25 mm jaws with DeBakey serration

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Endoscopic Bulldog Clamp  
for bowel surgery  
45 mm jaws with DeBakey serration

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Endoscopic Bulldog Clamp  
for bowel surgery  
60 mm jaws with DeBakey serration

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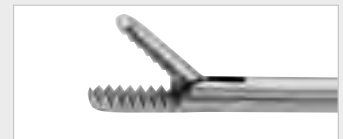
RZ Needle holder with axial handle,  
 LL-rinsing channel, ø 5 mm, length 33 cm

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RZ Needle holder with axial handle,  
 ø 5 mm, length 33 cm

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
RZ Needle holder with axial handle, with  
 tungsten carbide inserts, straight,  
 ø 5 mm, detachable, length 33 cm, with  
 LL-rinsing channel

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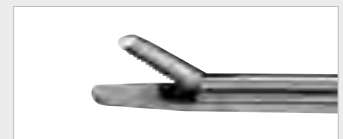RZ Needle holder insert only, straight

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RZ Needle holder with axial handle, with  
 tungsten carbide inserts, curved, ø 5 mm,  
 detachable, length 33 cm, with LL-rinsing  
 channel

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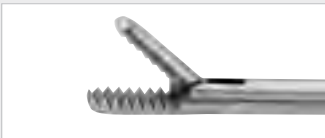
RZ Needle holder insert only, curved

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Needle Holder, with ratchet,  
LL-Rinsing channel, ø 5 mm

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Needle Holder, with ratchet,  
LL-Rinsing channel, ø 3 mm

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Needle Holder, with ratchet,  
jaw konkav-convex, ø 3.5 mm,  
with LL-rinsing channel

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Suturing Grasping Forceps, ø 5 mm. The jaws describe a U-shape. This enables stabilization of the wound edges at two separate points, enabling the penetration of the needle between the jaws preventing a risk of the entangling of the jaws, with LL-Rinsing channel

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Endo Needle holder with TC inserts, ø 5 mm, length 33 cm

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Endo-Needle holder ø 5 mm, 28 cm working length, with TC inserts

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Endo-Needle holder ø 5 mm, 28 cm working length, with TC inserts, curved left

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Endo-Needle holder ø 8 mm, 33 cm working length, with TC inserts

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Endo-Needle holder ø 10 mm, 33 cm working length, with TC inserts

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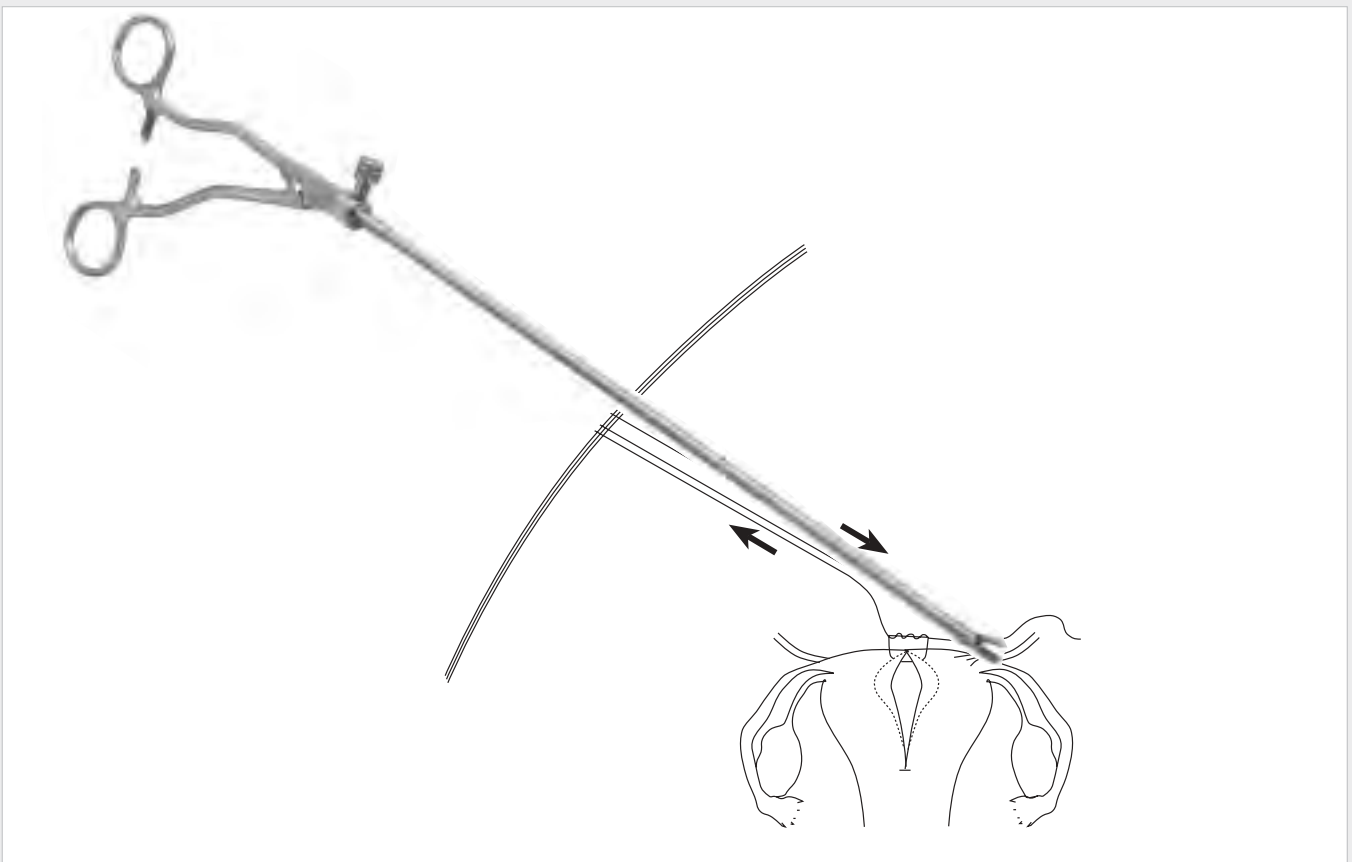
Tobias TC Needleholder with RZ handle and Luer-Lock connection, shaft length 36 cm, diameter 5 mm, jaw serrations with new geometry, TC insert in one jaw, for suturing and knotting in Minimally Invasive Surgery.

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

This illustration shows the (combined) endo/extracorporeal knotting technique:

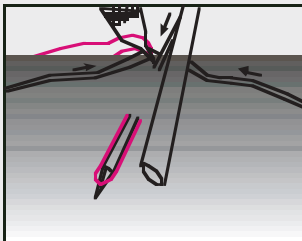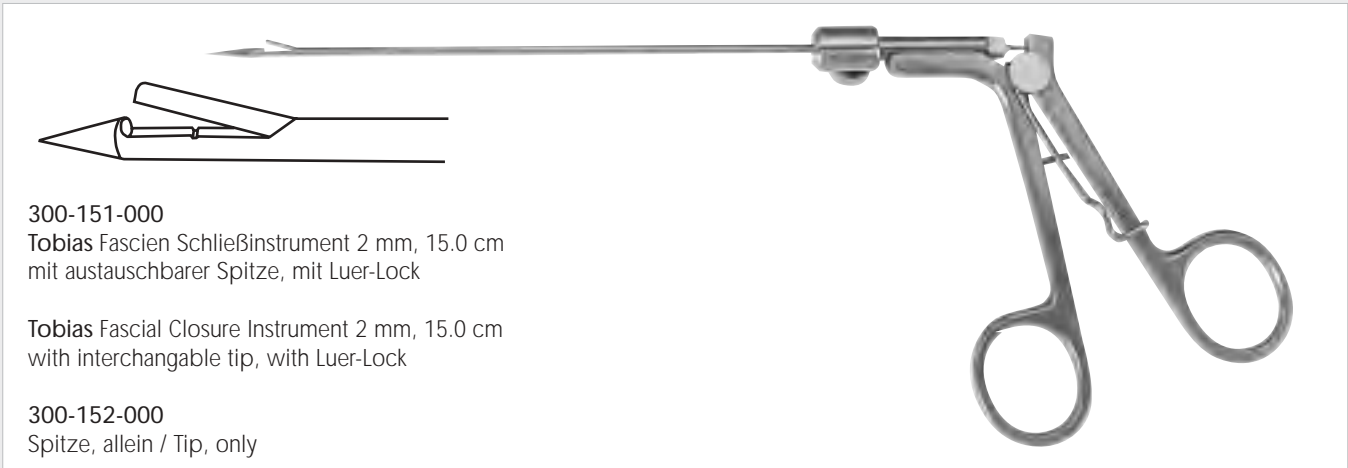
The thread is introduced into the abdominal cavity alongside the trocar. The endocorporal knot is prepared with a purse-string suture around the wound area. The short end of the thread is engaged and pulled. This allows a firm knot to be tightened and ensuring a good ligature.

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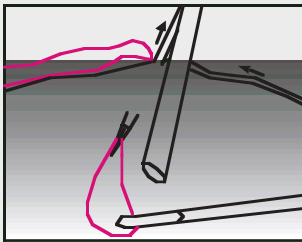
Grasp suture in mid-length with forceps. Under direct laparoscopic vision, with adequate pneumo-Peritoneum, place the Fascial Closure Instruments with suture into subcutaneous tissue directly adjacent to Trocar cannula. Incorporate all tissue layers as entry is made into the peritoneal cavity. Release suture.

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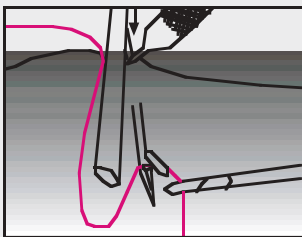
Remove instrument. Leave adequate loop of suture in the peritoneal cavity and adequate extra-corporeal falls.

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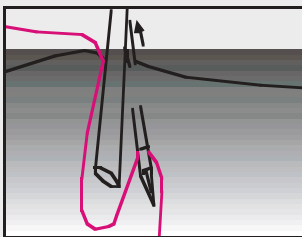
Reinsert Fascial Closure Instrument without suture, under direct vision, on opposite side of the trocar cannula. Place into the subcutaneous tissue incorporating all tissue layers. Grasp and retrieve the suture.

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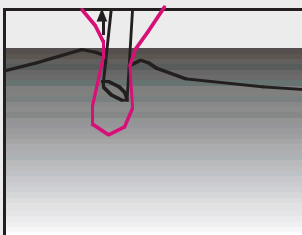
Remove the instrument. The forceps will hold suture securely upon removal.

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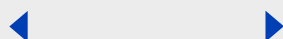Following closure of the last trocar, remove cannula. Tie suture in normal fashion. The Fascial Closure Instruments facilitates full thickness abdominal wall closure.

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
RZ-Knot tier for extracorporal knots

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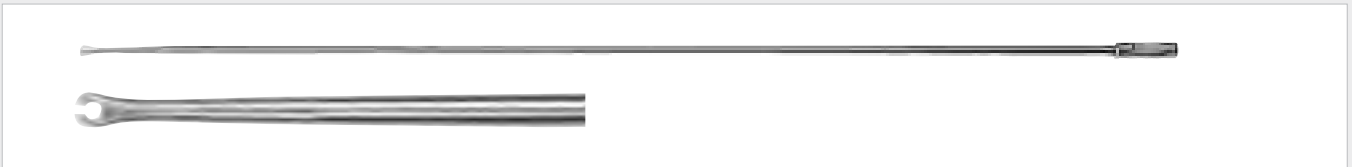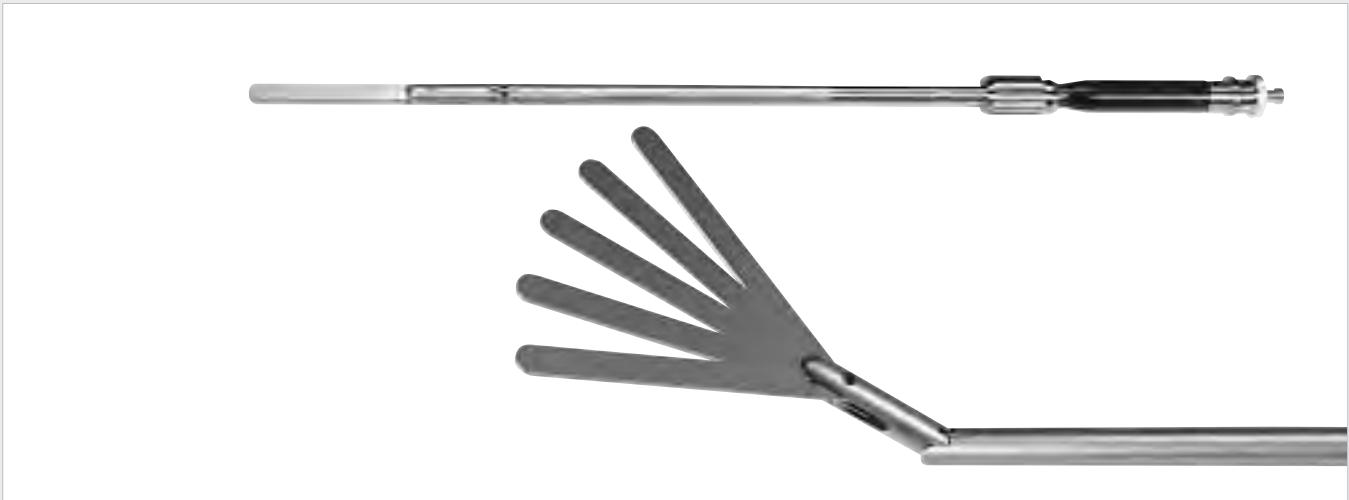
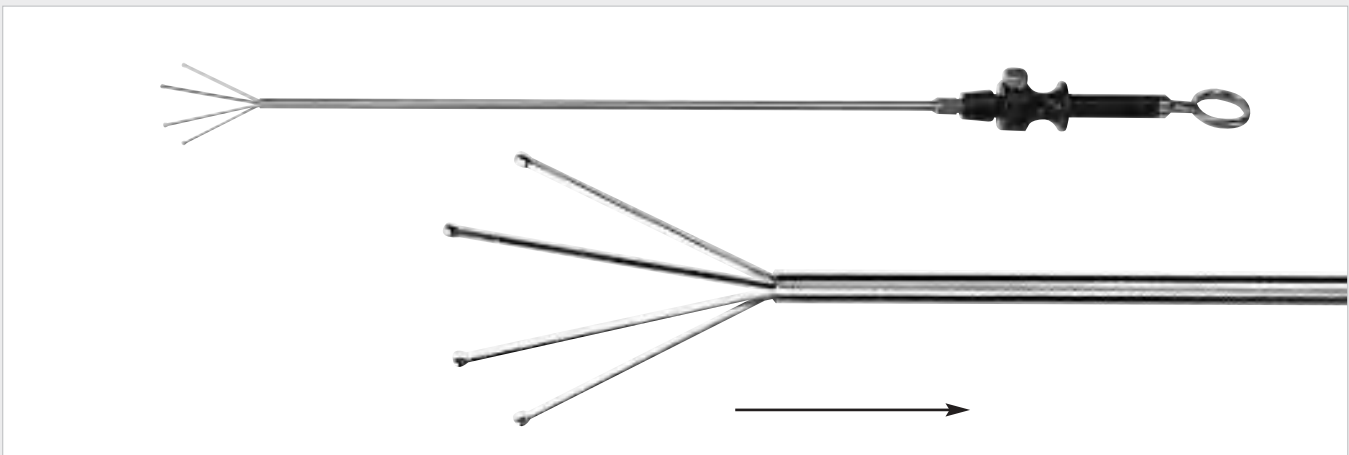
- Horizontal and vertical to 50° adjustable
- Softline fan design with wide tissue support
- Axial irrigation device with Luer-Lock-Adaptor
- Available with tube diameter 5 mm and 10 mm

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Bipolar curved scissors provides the  
ability of dissecting, cutting and bipolar  
coagulation

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### Detachable bipolar Coagulation Forceps

Reusable instrument for use in laparoscopy for grasping and cutting tissue during bipolar coagulation.

#### Generator Specifications:

This instrument has been designed for connection to the bipolar output of electrosurgical generators. The maximum power output shall not exceed 60 Watt..

### Comfortable Handling

The electrode can be rotated 360° for accurate jaw orientation. There is no receding of the jaw when closing. This allows precise grasping and coagulation of the tissue.

### Bipolar Scissors

The new, reusable bipolar curved scissors electrode for laparoscopy provides the ability to dissect, cut and coagulate at the same time. Changing the instrument is no longer necessary.

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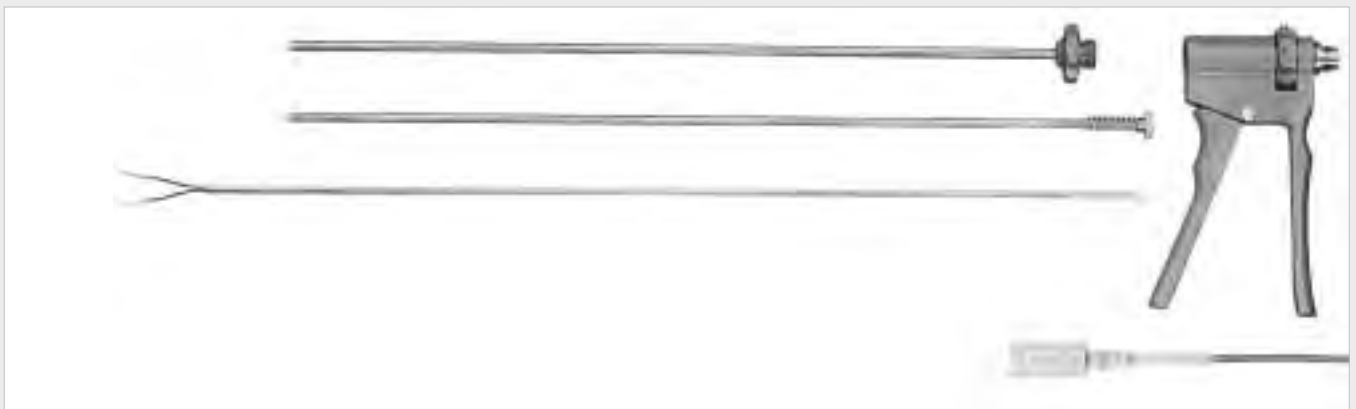
### Compatibility

The forceps in conjunction with the respective connection cable is compatible with the bipolar output of most electrosurgical generators not exceeding 60 Watt. An adapter can be used in order to connect a standard bipolar forceps cable.

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### Hygiene and Cost Effectiveness

The forceps is easily detachable in five parts. This allows to reliably clean even highly inaccessible areas. Because of its modular design, all parts are interchangeable and therefore easy to repair. The various electrodes can be all used in the same, ergonomically designed handle. The bipolar forceps can not be "flush" autoclaved.

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### Ring Handle

The new forceps body with ring handle, designed in cooperation with Dr. A. Scarfi, allows convenient and efficient working and is particularly useful when using the new bipolar scissors electrode. This ring handle is of course fully compatible with all electrodes, tubes and cables of the existing detachable forceps.

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**Detachable Bipolar Coagulation Forceps**

consisting of forceps body, inner- and outer  
tube - without electrode

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**Detachable Bipolar Coagulation Forceps**  
consisting of forceps body and shaft - without electrode

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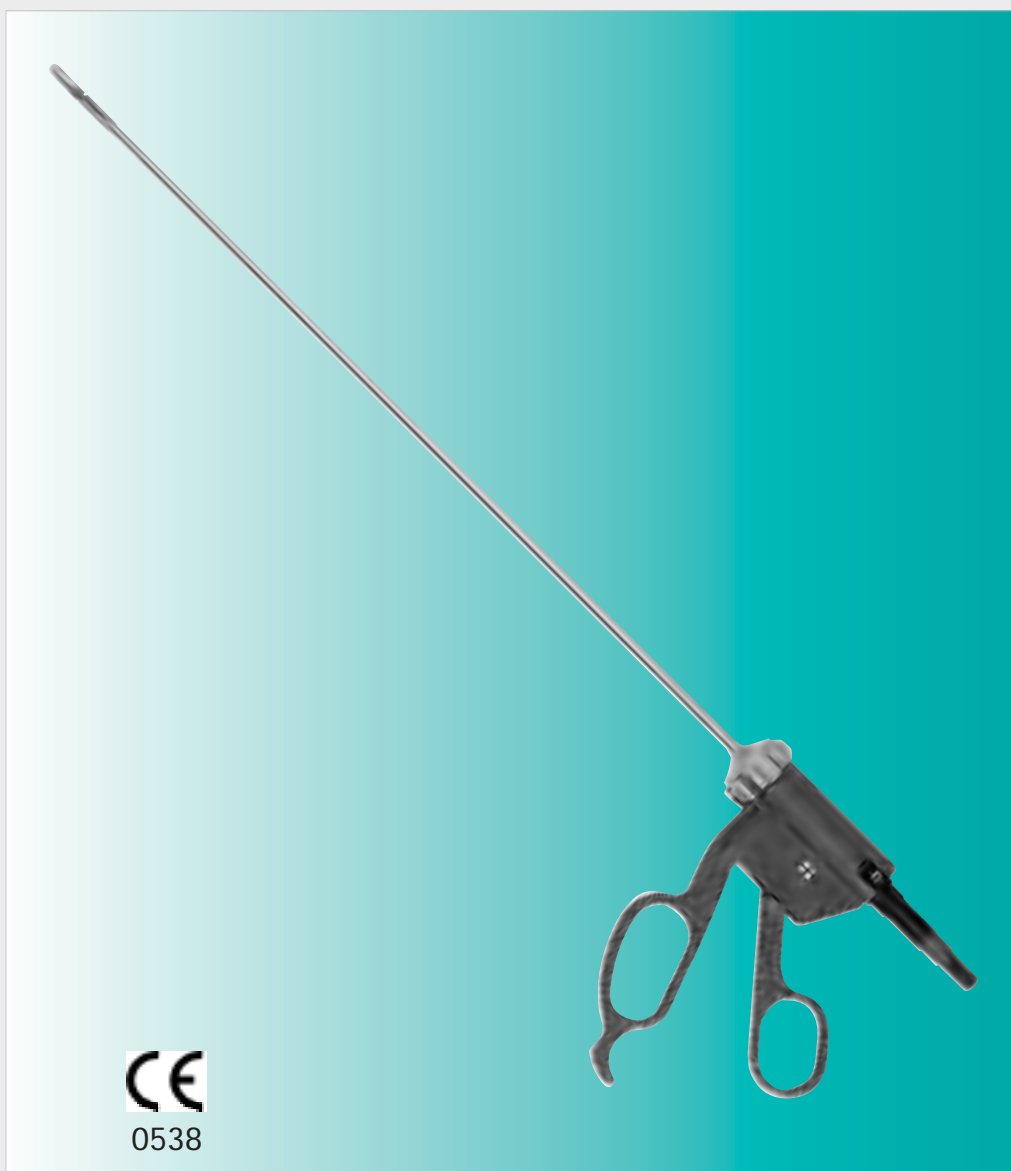
Maxi Grip Bipolar Forceps provides the ability of  
dissecting, cutting and bipolar coagulation

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Bipolar Coagulation forceps  
 ø 5 mm, length 33 cm and 45cm, detachable

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The forceps are detachable into 3 pieces -  
 handle - plastic tube - forceps insert/electrode

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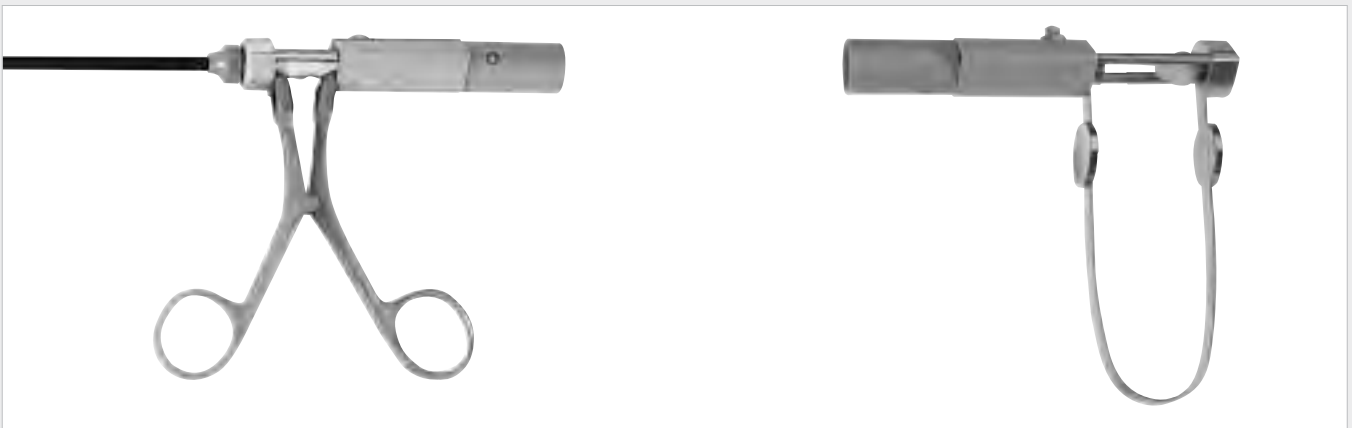
Bipolar Coagulation forceps  
 ø 5 mm, length 33 cm and 45 cm, detachable

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The forceps are detachable into 3 parts -  
 handle - plastic tube - forceps insert/electrode

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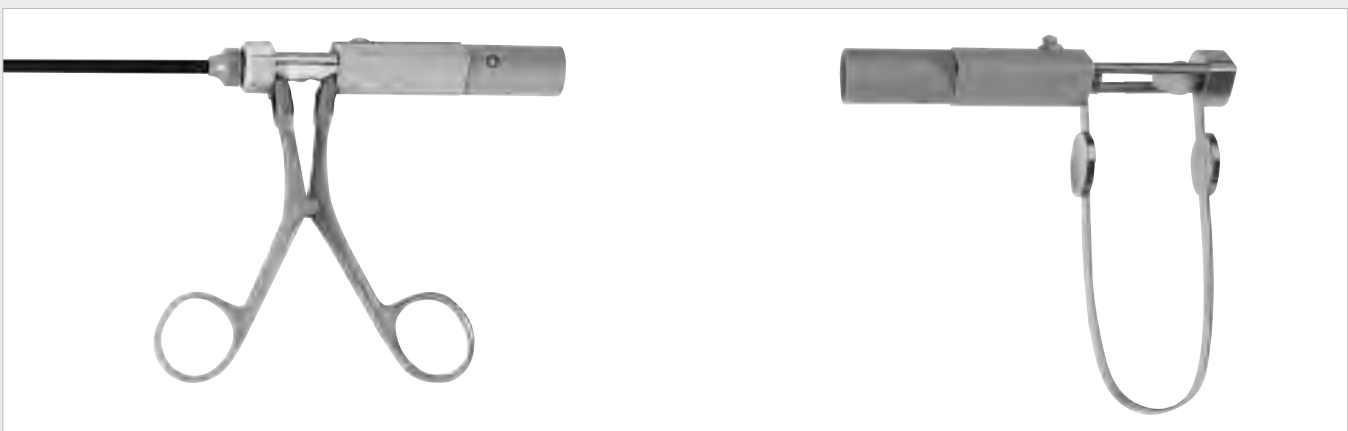
Bipolar Coagulation Electrodes

Bipolar coagulation electrodes for connection to the bipolar output of electrosurgical generators with dedicated parameters for bipolar coagulation.

The RZ Bipolar coagulation electrodes for endoscopy provide the ability to dissect and coagulate tissue. The program includes instruments for coagulation of larger areas as well as of small selective areas. Due to the special design of the electrodes, they are able to withstand high mechanical stress for safe handling with consistently good coagulation results. All instruments have a working length of 34 cm and 45 cm and a shaft diameter of 5 mm.

A slightly conical shaft end makes insertion into the trocar sleeve easier.

Using bipolar technology and the optimized configuration of the electrical poles, guarantees safe and gentle work with an exactly defined area of coagulation.

Due to the insulation between the electrodes, always the same distance and therefore constantly good coagulation results are achieved. Moreover, the electrodes are easier and more reliable to clean.


The ergonomically designed handle allows secure holding and easy rotation during work. The standard bipolar connector plug will fit conventional bipolar cables that are already available.

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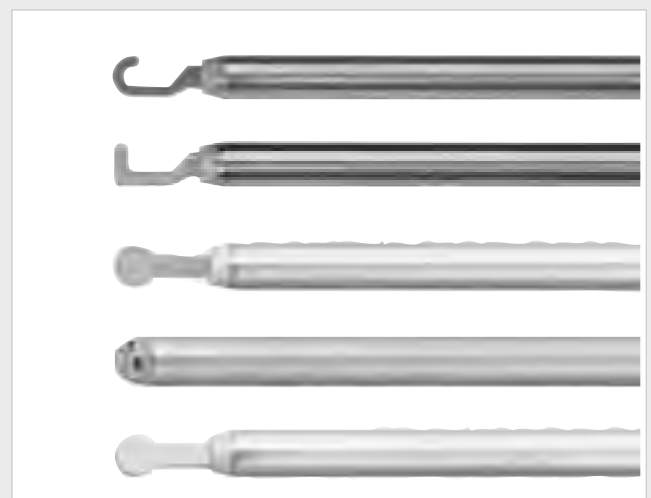Cable for bipolar Instruments, 3 or 5 m length

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**Special Features:**

- Double safety for patient: hardware and software controlled function to avoid overpressure
- Automatic and internal change over to a second gas bottle
- Hygiene guarantee: automatic gas flushing function, filtration of bacteria and other impurities
- LED Displays clearly indicate both preset and actual values guaranteeing easy observance
- Microcomputer controlled high-quality electronic and pneumatic components
- Numerous interfaces for data transmission to external devices

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**Insufflationsdaten / Insufflation Data:**

Gasfluß / Gas Flow: 0...20 l/minute 0...30 l/minute  
 Insufflationsdruck / Insufflationsdruck: 0...30 mm Hg (max. 50 mm Hg)

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The new Standard for Halogen Light Sources with  
Automatic Switch-over to the spare bulb in case of bulb failure. Easy handling, user friendly and well defined control panel, optimal hygienic conditions and easy to clean. Longer bulb lifetime up to 200\* hours

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The ALEXIS 180M XENON light source offers outstanding white examination light (sunlight spectrum) for all endoscopic applications, for example:

- during video endoscopy
- during fiber endoscopy
- during endoscopy using rigid optics
- during usage of forehead illumination

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#### Technical Data

Lamp type	XENON speciallamp type: XBO 180
Output / Performance	180 Watts
Lamp geometry	integrated ellipsoidreflector
Color temperature	5800° K (daylight spectrum)
Heat protection filter	IR 3 (Option KG/UV-Rejection filter)
Lifetime of XENON lamps	500 hours
Average value at	30 minutes ON and 15 minutes OFF
	No warranty on lamp!
Operating hours indicator	Digital
Power requirements	115 & 230 VAC,50/60 Hz, 3.15 / 1.6 Amp. +10% / -5%
Power consumption	350 VA
Fuses	2 x 3.15 AT, (2 x 1.6) IEC 127-2/III 5 x 20 mm
Dimensions (WxHxD)	32.5 x 26.5 x 14.5 cm
Weight	9.5 kg
Housing safety standard	protection class IPX 1 / IEC 529
	full metal casing, powder coated
Temperature - operation	+10° C - +40° C
Temperature - storage	-20° C - +45° C
Max. relative air humidity	85%
Radioscreening	No
Power supply cord	Euro / USA - Edition
Protection grade	MedGV Apparatus Group 1
Classification	Typ CF (cardiac floating)
Electr. protection class	1 / TÜV certified
CE standard	Council Directive 93/42/EEC Class I
Compliance	with Corresponds to IEC 601-2-18 : 1992 EN 60601-1 : 1993 EN 55011, IEC 801, TÜV Type tested
ESD 8EMV) in accordance to	DIN VDE 0751 Part 1

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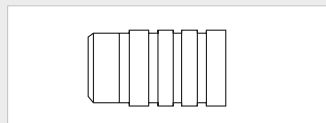
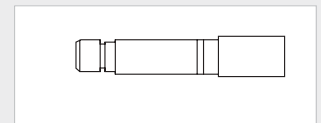
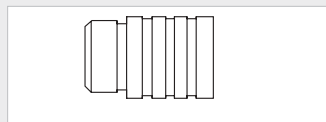
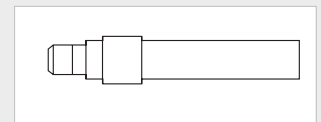
The RZ-410 is a waterproofed 1-Chip-Micro-colour camera with the technical features of a 3-Chip-Camera or endoscopy

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250-025-040 Zoom-Adapter von 25-40 mm  
Zoom Adaptor from 25-40 mm

#### Features:

- The Camera offers an **excellent image quality**- by using a new 1-Chip-video technology offering almost the same quality of a 3-Chip technology.
- Handling is simple and **user-friendly** without any complicated adjustments.
- User adjustments displayed on graphic display of the control unit.
- Automatic **memorizing of the configuration** when switched off.
- No help function on screen needed anymore.
- Decontamination by immersion (camera head)
- RZ-410 can be used for Halogene as well as for Xenon light sources

300-001-410 RZ-410 230V / PAL  
300-001-411 RZ-410 110V / NTSC

consisting of:

- 1 Control Unit
- 1 Camera head with 3.0 m cable
- 1 BNC-BNC cable
- 1 Y/C cable
- 1 Main cable
- 1 Operating manual
- 1 Focusing lens 28 mm

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**250-025-035**  
Fokussier Optik 35 mm  
Focusing lens 35 mm

## Technical data

Videosystem	PAL / NTSC
Image sensor	1/2" Interline Transfer CCD - Chip
Resolution	752(H) x 582(V) Pixel
Horizontal resolution	> 450 TV Linien at Y/C
Vertical resolution	> 420 TV Linien at Y/C
Sensitivity	2 Lux at F 1/1.3
White balance	Automatic 3200 Kelvin 6000 Kelvin Manual
AGC	Automatic / Off
Iris control	Automatic / Off
Exposure mode	Average light measuring Spot light measuring "Cut out"- Average light measuring "Cut out"- Spot light measuring
Video outputs (PAL)	2 x VBS signal to BNC- connector: 1Vpp / 75 W, Sync. negativ 2 x Y/C signal to S-VHS- connector: Y / 1,0Vpp / 75 WC / 0,3 Vpp / 75 W
Current consumption	0,065 Ampere
Power requirement	230V / 110V
Power frequency	50Hz
Fuses	2x T 80mA
Operating temperature	0°C to +40°C
Operating humidity	Up to 80 % (non condensation permissible)
Storage temperature	-20°C to +60°C
Storage humidity	Up to 90 % (non condensation permissible)
Dimensions	(W x H x D) 330 x 75 x 280 mm
Weight	3400g
Dimension Applied part	(Ø x L) Ø28.5 x 50 mm
Weight Applied part	240g including 3 m cable
Applied part fitting	C- mount- thread
Objective focal length	Variabel with TV- Adapter
Light source dependency	Suitable for Halogen and Xenon

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**Suction- and Irrigation Unit Set**, consisting of: Tubing Set with LL connections (300-001-511), 2 Ltr. glass aspiration bottle (300-001-512), puncture tube (300-001-514), 2 x bottle holder (300-001-515), vacuum lid (300-001-516), micro filter (300-001-518) and mains cable

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#### Aspirator / Irrigator for all surgical procedures

- Highest efficiency
- Aspirates and irrigates at up to 3 l/min
- Large two –liter aspiration container
- “Stand-by” function minimizes wear and tear
- Special filters ensure sterility
- Simple ergonomic operation
- Flasd may be attached to unit or carrier cart.
- No special adjustment necessary. Aspiration and irrigation are regulated and selected by the instrument controls
- Integrated overflow protection
- “Stand-by” function for low noise level
- High-quality components

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### 300-900-999

A complete Laparoscopy unit consisting of:

- Flat screen monitor 18" 300-001-118
- RZ 410 1 Chip Camera 300-001-410
- RZ Alexis 180 Xenon Light Source 300-001-180
- Continuous flow CO<sub>2</sub> insufflator 300-902-100
- Trolley 300-001-999

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### 300-001-999 Trolley

Dimensions: H: 1108 mm, W: 500 mm, D: 625 mm

A multifunctional system providing practical support and meets the requirement of a wide range of endoscopic procedures.

Equipped with various holding devices, the RZ-Mobile Unit Tower is a perfect system unit.

Stable trolley construction with integrated double support system

Disinfectant-proof epoxy resin powder coating

Ergonomically designed handle bar for easy and safe manoeuvring

Easy moving, large rollers, 125 mm

Integrated cable beam and sockets

Untippable construction on four anti-static wheels

All versions comply with MedGV, Group 3, class of protection as per VDE 0750, IEC 601-1

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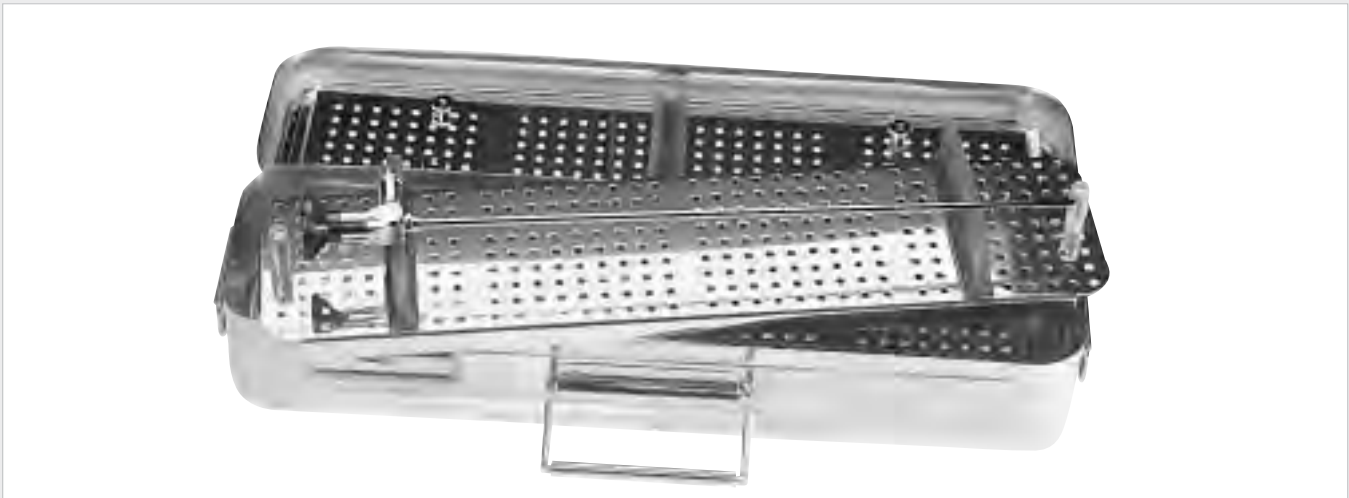
### Criteria for Cleaning Laparoscopic Instruments

- A. The laparoscopic instrument must be cleaned thoroughly immediately after use. All debris ( blood, tissue and secretions ) can be removed by rinsing the instrument under cold running water. Detachable instruments must be dismantled in accordance with our instructions.
- B. When manual cleaning of the instrument is required, use a soft cleaning tissue, with non-fluffy material.
- C. Instrument with a cleaning channel must be rinsed immediately after surgery with a suitable cleaning solution, by using an irrigating syringe.
- D. If a washing machine is to be used for the final cleaning of the instruments, remember to open the instrument fully and place in a wire basket separate from other instruments to avoid damage.
- E. In the event of a disinfecting solution being used on the instrument after washing; to avoid staining, immerse the instrument completely in fresh demineralized water for 30 minutes to neutralise any acids.
- F. Always dry the instrument thoroughly after the above procedures with non-fluffy soft material.

#### REMEMBER:

Sterilisation is not a substitute for cleanliness.

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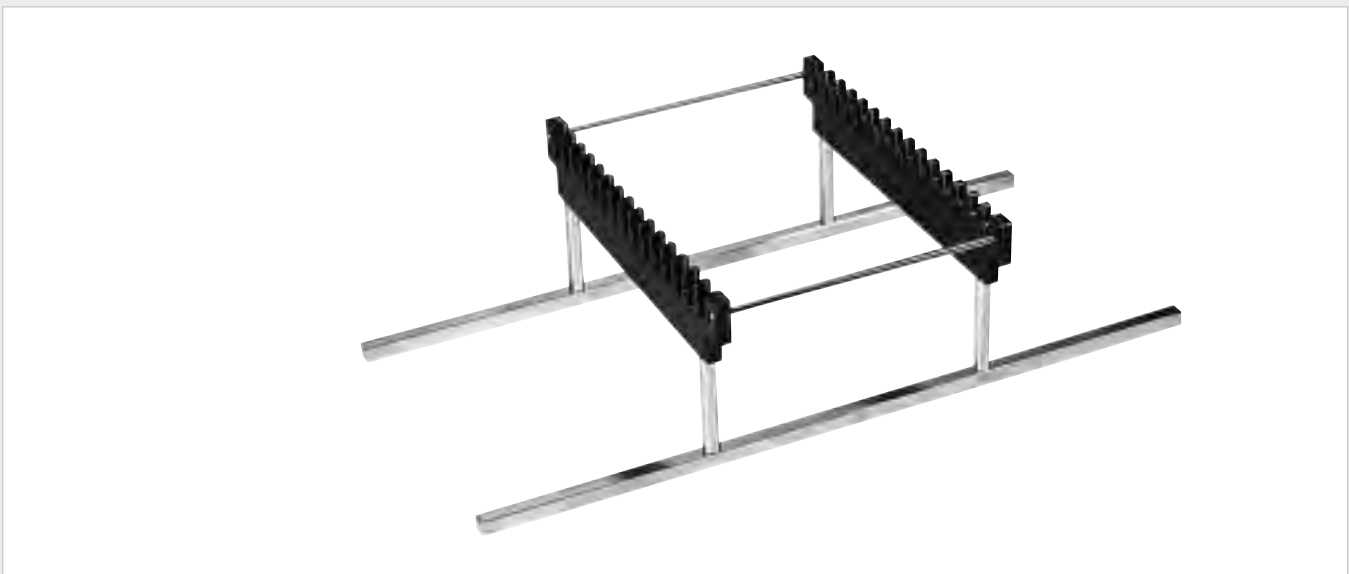
Storage and sterilization rack for 15 pcs.  
of MIS-instruments  $\varnothing$  5 mm and 10 mm,  
170 mm x 250 mm x 125 mm

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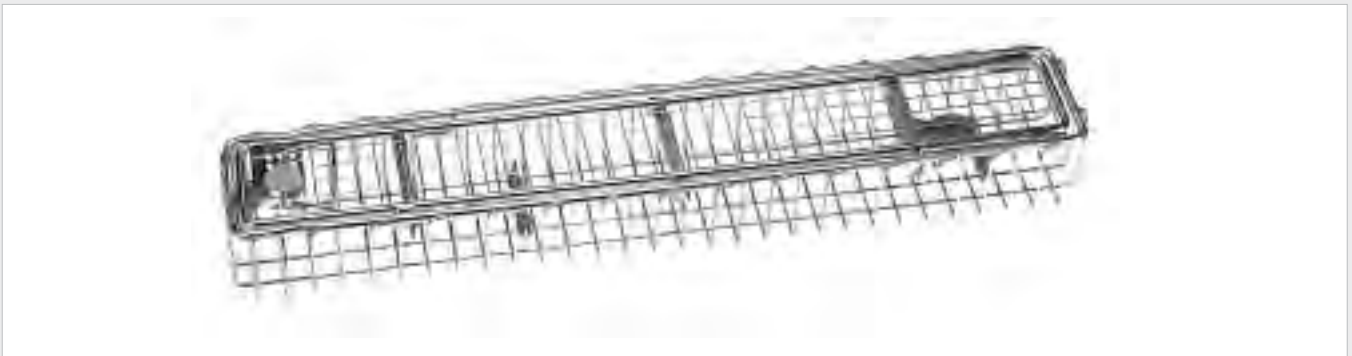
### Endoscope basket

Whether it is a matter of cleaning, sterilizing or storage and transport, the endoscope basket offers secure protection for the sensitive endoscopes against damage.

The perforated endoscope baskets are made of chrome nickel steel, are electrolytically polished, worked up without burr and have closed wire shovels.

The removable cover equipped with a special rotatable adapter protects against selfopening. Plugged on silicone store elements guarantee an optimal fixation of your endoscopes during water and sterilization process. The external basket is worked up without bear out edges and offer outstanding conditions for the bending of the baskets in foils. Beneficial for storage and transport of endoscopes after sterilization process.

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Endoscopic wire basket with cover, for arthroscopes and sinusscopes  
dimensions: 290 mm x 80 mm x 52 mm

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Endoscope basket with cover, for laparoscopes, cystoscopes, hysteroscopes etc.  
dimensions: 460 mm x 80 mm x 52 mm

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Cleaning brush for  $\varnothing$  5 mm, 50 cm length,  
(10 pcs. per box)

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Cleaning brush for  $\varnothing$  10 mm  
50 cm lang, (10 pcs. per box)

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Lubricating oil for instrument care 400 ml

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To reach a long life expectancy of telescopes it is necessary to follow these rules:

#### 1. GENERAL ADVISES

- This care instructions are valid for all types of RZ-telescopes.
- Always handle the telescope by grasping the body part or eyepiece.
- Store the telescope by using a proper container to protect it.
- Avoid strong mechanical influences (bending, falling down, etc.) on the telescope (glass lenses may break), especially on the objective- and ocular lens.

#### 2. CLEANING

- Soak the telescope in an enzymatic cleaner (e.g. Klenzyme) for approx. 3 min. and rinse it afterwards under clear water.
- Clear the telescope manually using a soft brush. Attention while cleaning the optical portions.
- Rinse with distilled or demineralized water.
- Dry thoroughly with a soft cloth.

#### 3. DISINFECTION

- Clean the telescope as per 2. before disinfection.
- Insert telescope in solution (Cidex, Steris). Do not allow more than 30 minutes in any solution, including sterile water.

**Attention: Use the disinfectant according to the solution-manufacturers' instructions!**

#### 4. STERILISATION

##### Gas sterilization (ETO)

- Clean the telescope as per 2. before sterilisation.
- Sterilise the telescope in the gas-sterilisation.

**Attention: Follow the instructions of the steriliser manufacturer!**

##### Steam sterilisation

**(Attention: Only for autoclavable telescopes!)**

- Clean the telescope regarding 2. before sterilisation.
- Sterilise the telescope in the autoclavable regarding the following conditions:  
Pressure: max. 2 bar  
Time: max. 8 min at 134°C/273°F(±5°)

**Attention: Use the steriliser according to the autoclave manufacturers' instructions. Do not perform additional cooling on the device after sterilisation (selfcooling)!**

##### Hot air sterilisation

Is not recommended for RZ-telescopes.

Please follow any further instructions pertaining to your particular steriliser and have it inspected from time to time.

#### 5. REPAIR SERVICE

RZ-Medizintechnik does the repairs of our own telescopes as well as of other brand scopes.

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RZ-Aquajet

Water Jet Pistol compl. with 9 tips 600-100-001 to 600-100-009 and 1 m pressure tubing, with strap and suspension hook

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
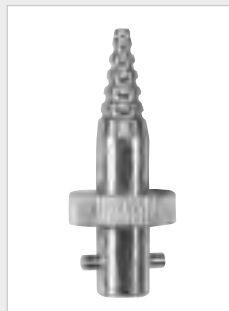
Cleansing Tip for Cones  
of Record Syringes

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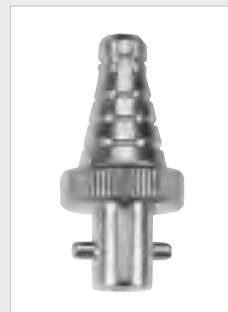
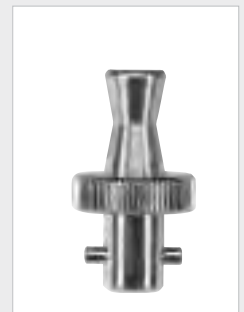
Cleansing Tip for  
Drainage Tubings

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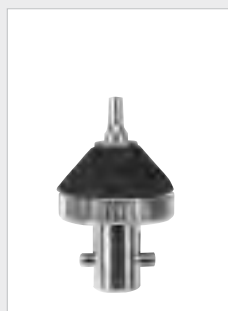
Cleansing Tip for  
Receptacles (Shower)

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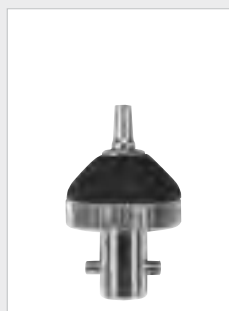Cleansing Tip for  
hypodermic needles  
with Record Cone

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
Cleansing Tip for  
hypodermic needles  
with Luer Cone

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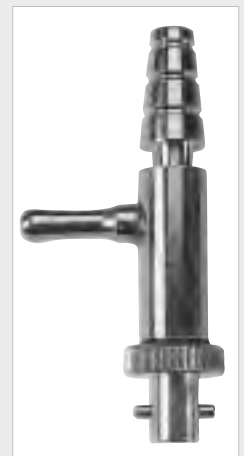Cleansing Tip for  
measuring and  
Volumetric Pipettes

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Cleansing Tip for  
(Water Jet Filter Pump)  
for fine Pipettes